DeltaFil Conditioner

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English

Product description

DeltaFil Conditioner is an aqueous polyacrylic acid solution for treating the smear layer following preparation and for conditioning the cavity, which optimizes the bonding of DeltaFil to the tooth's structure.

Intended purpose

- Treatment of the smear layer
- Conditioning of the cavity

Limitations of use

Application exclusively in combination with DeltaFil

Indications

- Loss of tooth substance due to caries, fractures, tooth wear or developmental disorders
- Secondary prevention of caries

Contraindications

- Do not use the material where there are known allergies to any of the components or contact allergies.

Patient target group

Persons receiving treatment as part of a dental procedure

Intended users

Dentist

Notes for use

- Carefully reseal the bottle after use and store away from light.

Recommended use

Pour DeltaFil Conditioner into a mixing well or onto a mixing block. Pick up the DeltaFil Conditioner using a cotton pellet or mini applicator sponge, apply it to the prepared surface and leave to work for 10 s. Then rinse thoroughly with water. Remove any excess water (e.g. with a cotton pellet or by blow drying gently). The prepared surface should still look damp, ready for further treatment. Do not over-dry the hard tissue!

Interactions

DeltaFil Conditioner should not be applied on acid-soluble linings.

Residual risks/side effects

- There are no known side effects to date. The residual risk of hypersensitivity to components of the material cannot be ruled out.

- Contact of the DeltaFil Conditioner with the mucous membrane can cause skin irritation. Rinse the affected area thoroughly with water.

Warnings / precautions

- Keep out of the reach of children!
- Avoid contact with skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid contact with the eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a doctor if necessary.
- Follow the manufacturer's instructions for other products that are used with the material(s).
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Product performance characteristics

Improves adhesive bond between tooth tissue and DeltaFil.

Storage / disposal

- Store at 15–25 °C (59–77 °F) in a dry, dark place!
- Do not use after the expiration date!

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