# DeltaFil

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#### English

#### Product description

DeltaFil is a radiopaque, classic glass ionomer cement for restorations in capsule form. DeltaFil chemically bonds to enamel and dentine and releases fluoride ions. It is available in white, dark and in the shades A1, A2, A3 and A4, based on the Vita shade system. Each capsule contains a minimum usable amount of 0.13 ml.

#### Intended purpose

- Permanent restorations class I, II: areas that are not subject to masticatory force; also suitable for the sandwich technique
- Permanent restorations class V; also for the sandwich technique
- Temporary restorations class I, II, III and V
- Extended fissure sealing
- Core build-up
- Base/liner
- Restorations on milk teeth class I, II, III and V

#### Limitations of use

The application is contraindicated if a relatively dry working area or the use of recommended application technique is not possible.

#### Indication

- Loss of tooth substance due to caries, fractures, tooth wear or developmental disorders
- Secondary prevention of caries

#### Contraindication

- Do not use directly on the exposed pulp.
- Do not use the material where there are known allergies to any of the components or contact allergies.

#### Patient target group

Persons receiving treatment as part of a dental procedure

#### Intended user

Dentist

#### Notes for use

- Clean instruments with water immediately after use.
- Observe the manufacturer's instructions for all materials that are used with the product.
- If the material is applied to the patient's mouth with a single use applicator, it must be used for this one patient only for hygienic reasons.
- For deep cavities, protect areas close to the pulp with a suitable liner, e.g. by using a small quantity of a firm-binding calcium hydroxide preparation.
- The time between activating the Applicap, mixing and application should be kept short to avoid premature curing of the material.
- During application, the tip of the capsule should be kept in the material to avoid possible air bubbles.
- Do not use DeltaFil without the DeltaFil Conditioner.
- For an optimal result, use the Applicap Dispenser from DMG.

#### Using the Applicap (for image, see » Handling«)

##### Activating and mixing the Applicap

- Shake the powder inside the Applicap before inserting it in the Applicap Dispenser. Do not activate it on the table surface! (Fig. 1).
- Insert the Applicap, groove first, into the holder of the Applicap Dispenser (Fig. 2) and activate it by firmly (or quickly) pressing the Applicap Dispenser lever once. The retaining ring must come off (Fig. 3).
- Remove the Applicap (Fig. 4) and insert it into the mixer fork of a capsule mixer (e.g. CapMix, Silamat S6 or Rotomix) and mix at 4000 to 4500 rpm for 10 s, or in the Rotomix for 8 s (Fig. 5).
- Reinsert the capsule into the Applicap Dispenser and remove the cap (Fig. 6). The material is dispersed by repeated pressing and can be applied directly. Apply within 10 seconds (Fig. 7).
- After pressing the reset button, the Applicap can be removed from the Applicap Dispenser (Fig. 8).

##### Recommended use

- Prepare the tooth as usual. The cavity should be shaped in such a way that the material layer is no thinner than 0.5 mm at any point.
- Next, rinse the cavity thoroughly with water, removing any excess water (e.g. with a cotton pellet or by blow drying gently). The prepared surface should still be slightly damp. The dentine must not become too dry.
- Protect the areas close to the pulp.
- How to use DeltaFil Conditioner: Using a cotton pellet or mini applicator sponge, apply DeltaFil Conditioner to the prepared surface and leave to work for 10 s. Then rinse thoroughly with water. Remove any excess water as described above. The surface should still look moist, ready for the next steps.

**Note:** DeltaFil Conditioner is an aqueous polyacrylic acid solution for the treatment of the smear layer following preparation and for the conditioning of enamel and dentin for optimized bonding. It is applied as described above. Carefully reseal the DeltaFil Conditioner bottle after use.

- Use a matrix if necessary.
- Insert the mixed material directly into the cavity from the Applicap, compact and adapt to the margins of the cavity before modeling the surface. DeltaFil is workable for 120 s from the start of mixing.
- During application and modeling, avoid contamination of the surface of the glass ionomer cement with saliva or water.
- Then allow the material to cure for approx. 240 s.
- After 360 s from the start of mixing, finish the restoration as usual, e.g. with abrasive stones and finishing burs. Milling can be carried out with or without the use of water spray.

### Disinfection of the Applicap Dispenser

- The Applicap Dispenser can be reused after disinfection, although they should be replaced if there are visible signs of wear.
- A commercially available surface disinfectant is sufficient for disinfection purposes. DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch-Institute).

### Timing

Mixing time	10 s CapMix/ Silamat or 8 s Rotomix
Apply capsule contents after mixing.	≤ 10 s
Working time (23 °C, 73 °F) from the start of mixing *	≤ 120 s
Net setting time (37 °C, 98 °F)	≥ 240 s
Start time for finishing, from the start of mixing	≥ 360 s

\* The indicated times are valid for a room temperature of 23 °C/73 °F and a normal relative humidity of 50%. Higher temperatures shorten these times and lower temperatures lengthen them.

### Interactions

No interactions are currently known.

### Residual risks/side effects

There are no known side effects to date. The residual risk of hypersensitivity to components of the material cannot be ruled out.

### Warnings/precautions

- Keep out of the reach of children!
- Avoid contact with skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid contact with the eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a doctor if necessary.
- Follow the manufacturer's instructions for other products that are used with the material(s).
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

### Product performance characteristics

Complies with ISO 9917-1:2007 chemical type C and use type C

### Storage/disposal

- Store at 15–25 °C (59–77 °F) in a dry place!
- Do not use after the expiration date!

### Composition

Ionomer glass, polyacrylic acid, water, tartaric acid, additives.

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