Flairesse Gel

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English

Product description

Prophylaxis Gel for final treatment as part of professional tooth cleaning (PTC) and for integration into an individual prevention plan. Flairesse Gel is available in three different flavours.

Intended purpose

- Supports remineralization of enamel after professional tooth cleaning.
- Remineralization of initial caries.

Limitations of use

Do not use the material if the patient's control over their swallowing reflex is not guaranteed.

Indications

- Teeth with a high caries risk
- Incipient caries

Contraindications

Do not use the material where there are known allergies to any of the components or contact allergies.

Patient target group

People treated in the course of a dental procedure.

Intended users

Dentist / dental personnel

Notes for use

- If the material is applied into the patient's mouth with a single use applicator, the single use applicator must be used for this one patient only for hygienic reasons.
- The bottle is fitted with a tamper-proof seal. Make sure that the tamper-proof seal is intact before first use.
- When using the bottle for the first time, unscrew the cap, peel off the tamper-proof seal using the tag and discard.
- Shake well before each use!

Recommended use

Caution! Risk of swallowing if saliva suction is insufficient.

- ▶ Saliva must be suctioned away thoroughly during treatment.

Apply the gel with a polishing cup

1. Open the bottle cap.
2. Hold the bottle vertically with the opening pointing down and express the gel into a dampen glass by gently squeezing the bottle.
3. Dry the tooth surfaces.
4. To fill, hold the idle polishing cup in the Prophylaxis Gel and pick up the material.

Note: Any commercially available polishing cup may be used.

5. Work in the Prophylaxis Gel using the polishing cup.
6. Get the patient to spit out the excess Prophylaxis Gel.

Caution! Incorrect use may result in overheating.

- ▶ Ensure that there is sufficient material in the polishing cup used.

Note: For guaranteed effectiveness, inform the patient that they should not do the following for 30 minutes after using the prophylaxis gel:

- eat
- drink
- rinse their mouth

Applying the gel with an application tray

1. Open the bottle cap.
2. Hold the bottle vertically with the opening pointing down and express the gel into the application tray by gently squeezing the bottle.
3. Dry the tooth surfaces.
4. Apply the application tray to the dental arch.
5. Leave the application tray on the dental arch for 1 minute.
6. Remove the application tray at the end of the response time.
7. Get the patient to spit out the excess Prophylaxis Gel.

Note: For guaranteed effectiveness, inform the patient that they should not do the following for 30 minutes after using the prophylaxis gel:

- eat
- drink
- rinse the mouth

Interactions

No interactions are currently known.

Residual risks/side effects

- Swallowing excessive amounts of the material may cause nausea.
- The residual risk of hypersensitivity to components of the material cannot be ruled out.

Warnings / precautions

- For dental use only.
- Keep out of the reach of children!
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse the eyes thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.
- Do not swallow.

Performance characteristics of the device

Fluoride content: approx. 12,300 ppm

Storage / disposal

- Store in a dry place at 20 to 25°C (68 to 77°F)!
- Do not store in the refrigerator.
- Keep away from direct sunlight.
- Do not use after the expiration date.
- Dispose of in accordance with national regulations.

Composition

Water (approx. 88 %), sodium fluoride (< 3 %), phosphoric acid (< 3 %), additives (approx. 7 %)
Free from aspartame, gluten and saccharin.
Contains xylitol.

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