Honigum Pro Light

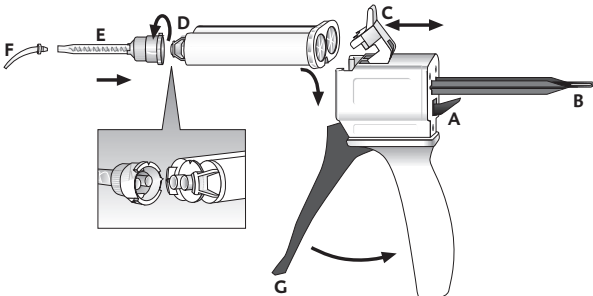
DMG Chemisch-Pharmazeutische Fabrik GmbH
Elbgaustraße 248 · 22547 Hamburg · Germany · www.dmg-dental.com
092221/2025-08

CE 0482

Handling

Using the Automix Cartridge

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Deutsch

Produktbeschreibung

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Honigum Pro Light: Polyvinylsiloxan, SiO₂, Polydimethylsiloxan, Methyl hydrogen polysiloxan, Aluminiumsilikat, Additive, Pigmente, Katalysatoren.

Honigum Pro Light Fast: Polyvinylsiloxan, SiO₂, Polydimethylsiloxan, Methyl hydrogen polysiloxan, Aluminiumsilikat, Additive, Pigmente, Katalysatoren.

Instructions for use

English

Product description

Honigum Pro Light is a correction impression material based on A-silicones. The material is available in the Automix Cartridge.

Intended purpose

Impressions for the fabrication of indirect restorations, such as crowns, bridges, inlays and onlays.

Limitations of use

Do not use the material for monophasic impressions.

Indication

All dental situations requiring final impressions, e.g. tooth loss, caries and trauma.

Contraindication

Do not use the material where there are known allergies to any of the components or contact allergies.

Patient target group

Persons receiving treatment as part of a dental procedure.

Intended user

Dentist, dental assistant staff

Notes for use

- The recommended times can be found in the section headed »Timing«.
- If the material is applied to the patient’s mouth with a single use applicator, the single use applicator must be used only for this one patient for hygienic reasons.
- In order to achieve an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are available as refills.
- The Intraoral Tips are to be used for direct, intra-sulcular application around the preparation.
- In order to improve the adhesion of the initial impression material to metal or plastic trays, a tray adhesive for A-silicones should be applied (e.g. DMG Tray Adhesive).

Using the Automix Cartridge (Fig. see “Handling”)

Using the Automix Cartridge

1. Push up the lever [A] at the back of the dispensing gun and pull back the slide [B] completely.
2. Lift up the plastic lever [C], insert the Automix Cartridge and lock it using the plastic lever [C].

Note: Make sure that the notches on the Automix Cartridge line up with those on the dispensing unit.

Attaching a mixing tip

1. Remove the cartridge cap [D] or used mixing tip after turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

Note: Make sure that the notches on the mixing tip line up with those on the Automix Cartridge.

3. Twist the mixing tip clockwise by 90° to lock it.
4. Attach an Intraoral Tip [F], if applicable.

Applying the material

- When using an Automix Cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- Mix the material in the mixing tip using the lever [G] on the dispensing gun and apply directly.

Note: After the application, leave the used mixing tip on the Automix Cartridge as a seal!

Removing the Automix Cartridge

1. Push up the lever [A] at the back of the dispensing gun and pull back the slide [B] completely.
2. Lift up the plastic lever [C] and remove the cartridge.

Disinfecting the Automix Dispenser

The Automix Dispenser can be reused after disinfection, although it should be replaced if there are visible signs of wear. Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch Institute). The Automix Dispenser is also autoclavable.

Recommended use

Double mixing impression

1. Fill the impression tray with Honigum Pro Putty, Honigum Pro Heavy or Honigum Pro Mono. When doing so, hold the impression tray at a slight angle beneath the mixing tip. Follow the corresponding instructions for use!

Note: To avoid air bubbles, the end of the mixing tip should remain immersed in the material throughout application.

2. Inject Honigum Pro Light around the preparation.

Note: Filling of the impression tray and injection should be concluded at the same time, as the higher temperature in the mouth causes the injected material to set more quickly than the material in the impression tray.

3. Position the filled impression tray slowly and gently in the mouth within the working time and take the impression of the preparation.
4. Remove the impression from the mouth once the setting time has ended (see “Removal”).

Corrective impression

Note: Do not allow more than 30 min to pass between completion of the initial impression and the final impression and do not clean the impression with alcohol, as there is then no guarantee of a secure bond between the two impression materials.

1. Cut generous drainage channels into the initial impression for the wash and remove imperfections.
2. After testing by repositioning, rinse the impression thoroughly with water and then dry.
3. Apply Honigum Pro Light around the preparation and onto the initial impression.
4. Position the filled impression tray slowly and gently in the mouth within the working time, initially exerting brief pressure.
5. Remove the impression from the mouth once the setting time has ended (see “Removal”).

Removal

1. In the area where the teeth have been prepared, remove the impression parallel to the longitudinal axis of the teeth.
2. After removal, proceed as described in the section “Cleaning and disinfection of the final impression.”

Cleaning and disinfection of the final impression

- Only disinfect the final impression using dipping solutions.
- DMG recommends the use of a 2 % glutaraldehyde solution.
- Ensure that the disinfectant used is indicated for use with addition-cured silicone impression material.
- Follow the manufacturer’s instructions for the disinfectant solution precisely!
- Leave the final impression to dry and store until casting (see “Storage of the impression”).
- For more information, see the statement issued by the German Society for Dental and Oral Medicine (DGZMK). Comments of the DGZMK 3/93 V 1.0; www.dgzmk.de
- ADA recommends following the advice of the Council on Dental Materials, Instruments and Equipment. JADA 1991;122(3):110; www.ada.org

Casting and galvanization

- After removing the impression from the mouth, wait at least 30 min before casting.
- DMG recommends using a special purpose hard plaster (ISO 6873:2013; Type 3, e.g. Moldano, Bayer) for the cast and an ultra hard dental plaster for the models (ISO 6873:2013; Type 4, e.g. Fujirock II, GC). The impression can be copper or silver-plated.

Storing the impression

- Store the impression at 15 – 25 °C (59 – 77 °F) in a dry place!
- Protect from exposure to direct sunlight!
- The impression is stable and can be cast even after months without loss of accuracy.

Timing

Honigum Pro Light		
		Fast
Working time including mixing time* [s]	≤ 165	≤ 75
Setting time in the mouth [s]	≥ 195	≥ 120

* The specified times are valid at a room temperature of 23 °C and a normal relative humidity of 50 %. Higher temperatures shorten, lower temperatures lengthen these times. The working time describes the period from when mixing begins to the time by which the impression tray must be positioned in the patient’s mouth.

Interactions

- Rinse off any residual retraction solution thoroughly with water, otherwise this may lengthen the setting time in some instances.
- Never allow the impression to come into contact with organic solvents or liquids containing solvents as this may result in a swelling of the material and may therefore lead to inaccuracies in the cast.
- Do not allow the impression material to come into contact with latex gloves. DMG recommends the use of PVC or PE-based gloves.
- Do not combine the material with condensation-curing silicone or polyether impression materials.
- Leftover methacrylate-based materials may alter the setting behavior of the silicone impression materials. If a restoration was prepared beforehand, carefully clean the preparation and the surrounding tissue with alcohol-soaked gauze and water spray before taking the impression.

Residual risks/side effects

- We place great importance on the quality of our products. However, please check the product for damage and visible contamination before use.
- There are no known side effects to date.
- No cases of hypersensitivity or allergic reactions are known to date. However, reactions to components of the material cannot be ruled out.

Warnings/precautions

- For dental use only!
- Avoid contact with the eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer’s instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer (productsafety@mtcompanies.com) and to the responsible registration authorities.

Product performance characteristics

Meets the requirements of ISO 4823:2021, Type 3: light-bodied consistency

Storage/disposal

- Store in a dry place at 15 to 25 °C (59 to 77 °F).
- Protect from exposure to direct sunlight!
- After the application, leave the used mixing tip on the Automix Cartridge as a seal!
- Do not use after the expiration date!
- Disposal must comply with national regulations.

Composition

Honigum Pro Light: Polyvinylsiloxane, SiO₂, polydimethylsiloxane, methyl hydrogen polysiloxane, aluminum silicate, additives, pigments, catalysts.

Honigum Pro Light Fast: Polyvinylsiloxane, SiO₂, polydimethylsiloxane, methyl hydrogen polysiloxane, aluminum silicate, additives, pigments, catalysts.

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