LuxaBite

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SiO₂, EBPADMA, Urethanmethacrylat, Polyester, Additive.

Instructions for use

English

Product description

LuxaBite is a bis-acrylate-based self-mixing bite registration material. The material presents only negligible resistance to the closure of the patient's jaw. Accordingly, a reliable bite registration of the occlusion is guaranteed. A detailed recording of the occlusal relief and the high final hardness simplify the precise positioning of working and counter-bite models.

Intended purpose

Bite registration

Limitations of use

Do not apply LuxaBite directly to oral mucosa in edentulous areas as heat may develop.

Indication

Dental cases that require impressions of both jaws, e.g. tooth loss

Contraindication

Do not use the material if there are known allergies to any of the components or contact allergies.

Patient target group

People who are receiving dental treatment.

Intended user

Dentist, dental assistant staff

Notes for use

- For the bite registration of complete dental arches, use LuxaBite in segments (3 – 4 cm).
- To achieve an optimum mixing result, use the mixing tips available from DMG. All mixing tips are also available in refill packs.

Handling the Automix Cartridge (Fig. see “Handling”)

Inserting the Automix Cartridge

1. Push up the lever [A] at the back of the Automix Dispenser and pull back the slide [B] all the way.
2. Lift up the plastic latch [C], insert the Automix Cartridge and lock it using the plastic latch [C].

Note: Make sure that the notches on the Automix Cartridge line up with those on the Automix Dispenser.

Attaching a mixing tip

1. Remove the cartridge cap [D] or used mixing tip after turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

Note: Note: Make sure that the notches on the mixing tip line up with those on the Automix Cartridge.

3. Turn the mixing tip 90° clockwise to lock it.

Applying the material

When using an Automix Cartridge for the first time, squeeze out a pea-sized amount of material and discard it.

1. Mix the material in the mixing tip using the lever [F] on the Automix Dispenser and apply directly.

Note: After use, clean the used mixing tip with a standard disinfectant solution and leave it on the Automix Cartridge to act as a cap!

Removing the Automix Cartridge

1. Push up the lever [A] at the back of the Automix Dispenser and pull back the slide [B] all the way.
2. Lift up the plastic latch [C] and remove the Automix Cartridge.

Disinfecting the Automix Dispenser

- The Automix Dispenser can be reused after disinfection, although it should be replaced if there are visible signs of wear.
- Disinfection can be carried out using a commercially available immersion disinfectant. DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch Institute).

Recommended use

Habitual bite registration

1. Lightly isolate any existing plastic components with Vaseline or a similar separating agent. Block out any existing bridge pontics using a suitable material, e.g. wax.
2. Apply LuxaBite to the dry dental arch. When doing so, the strand length of the material may be 3 to 4 cm. Prepared areas should be covered first.
3. Allow the patient to adopt the desired occlusion.
4. In order to ensure the complete bite registration of the occlusal surfaces, apply additional material under the buccal cusps of the upper jaw teeth.
5. Once the material has cured for 45 to 60 s, have the patient open their bite slightly and then close it once more. This will ease the subsequent removal of the registration.
6. Once the material has cured for a further 90 s, remove the material with gloved fingers or using a suitable instrument.
7. In order to avoid positioning errors, reduce the registration to the occlusal contact surfaces after final curing. To do so, use a suitable instrument, e.g. a plastic bur.
8. Articulate cast models using the bite registration.

Centric bite registration using an anterior jig

1. Use a suitable material, e.g. composite or silicone, for bite fixation. Place the material on the anterior lower jaw and guide the patient to the desired occlusion. The dental arches are not fully closed, in order for space to remain for the application of the bite registration material.
2. The procedure now continues as described under points 2 to 8 for habitual bite registration.

Disinfecting the bite registration

The bite registration can be disinfected using standard disinfectant solutions. The use of products containing alcohol is recommended because these additionally remove the oxygen inhibition layer.

Timing

Total working time from the start of mixing [s]	≤ 45
Patient: Open bite slightly and close again [s]	45 - 60
Remove registration from the mouth [s]	After a further 90
Final curing after removal [s]	240

Interactions

Avoid contact with retraction solutions as retraction solution residues can increase the setting time in some cases. Rinse away residues of retraction solutions thoroughly with water prior to the bite registration.

Residual risks/side effects

- Please check the product for damage and visible contamination before use.
- Hypersensitivity and/or allergic reactions cannot be ruled out in individuals who are predisposed.
- A low residual risk of unknown side effects cannot be ruled out.

Warnings/precautions

- For dental use only!
- Avoid contact with the skin! In the event of accidental contact with the skin, immediately wash the affected area thoroughly with soap and water.
- Avoid contact with the eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products used with the material/materials.
- Observe the safety data sheet.
- The manufacturer (productsafety@mtcompanies.com) and the responsible registration authorities must be notified of any serious incidents involving this product.

Product performance characteristics

Meets the requirements of ISO 4823:2025, Type B

Storage/disposal

- Keep away from direct sunlight.
- Store at 15 – 25 °C (59 – 77 °F) in a dry place.
- After use, clean the used mixing tip with a standard disinfectant solution and leave it on the Automix Cartridge to act as a cap!
- Do not use after the expiration date!
- Disposal must comply with national regulations.

Composition

SiO₂, EBPADMA, urethane methacrylate, polyester, additives.

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