DMG MiniDam

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Elastomeres Silikon (ca. 99 %), Pigmente, Additive

Product description

The gingival protection device is comprised of silicone and is used to facilitate a relatively dry environment in the proximal area between two teeth and to protect the gingiva against dental materials.

Intended purpose

Establishing a relatively dry environment in the presence of complete dentition, for example when using Icon or Composite filling materials.

Limitations of use

Not suitable for establishing absolutely dry environments.

Indications

- Caries
- Fluorosis
- Idiopathic or trauma induced spots
- Loss of tooth substance due to trauma, tooth abrasion or development disorders

Contraindications

Do not use the material where there are known allergies to any of the components or contact allergies.

Patient target group

People treated in the course of a dental procedure.

Intended users

Dentist

Notes for use

- The DMG MiniDam is for single use and for hygienic reasons can only be used on one patient.
- Avoid damaging the gingival protection device with dental floss or other materials. Do not use damaged gingival protection devices.

Recommended use

1. Clean the working area.
2. If the teeth are very close together: separate the teeth using a wedge.
3. Pre-stretch the gingival protection device by gently pulling the tabs apart.
4. Position the gingival protection device over the affected teeth. Make sure that the tabs are pointing in the direction of the area being worked on (Fig. 1).
5. Pull the tabs apart (Fig. 2).
6. Place the interdental section of the gingival protection device in the proximal position.

Note: The middle interdental section is used to protect the working area.

7. Pull the gingival protection device towards the gingiva until the interdental sections are positioned in the proximal area (Fig 3).
8. Release the gingival protection device (Fig. 4).
9. Fold the gingival protection device cervically where the teeth are isolated.
10. Fold the tabs cervically (Fig. 5).
11. Remove wedges as appropriate.
12. Verify that the gingival protection device is properly seated and correct if required (Fig. 6).
13. Further measures to facilitate a dry environment (e.g. cotton rolls, aspiration) can be carried out as required.
14. Perform treatment.
15. Remove the gingival protection device and discard.

Interactions

No interactions with other materials are currently known.

Residual risks/side effects

There are no known side effects to date. The residual risk of hypersensitivity to components of the material cannot be ruled out

Warnings / precautions

- Keep out of the reach of children!
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Performance characteristics of the device

- Tear strength ≥ 15 [N]
- Tensile strength ≥ 15 [mm]

Storage / disposal

- Store in a dry place at 15 to 25°C (59 to 77°F)
- Dispose of in accordance with national regulations.

Composition

Silicone elastomers (approx. 99%), pigments, additives

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