DMG Retraction Paste

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English

Product description

DMG Retraction Paste is a retraction medium for sulcus expansion and has an astringent effect on the gingiva.

Intended purpose

For temporary marginal gingival retraction and provision of a dry sulcus environment in the case of healthy periodontium.

Limitations of use

Do not use in the case of exposed bone, advanced periodontopathy or exposed furcation defects.

Indications

All dental cases that require final impressions of prepared teeth e.g. tooth loss, caries and trauma.

Contraindications

Do not use the material where there are known allergies to any of the components or contact allergies.

Patient target group

Persons receiving treatment as part of a dental procedure.

Intended users

Dentist

Notes for use

- If the material is applied to the patient's mouth with a single use applicator, the single use applicator must be used for this one patient only for hygienic reasons.
- Use the compule immediately after opening the aluminum bag. Throw away residual material in the compule.

Recommended use (for image, see »Handling«)

1. Prepare the tooth.
2. Place the compule in a suitable dispenser.
3. Bend the tip of the compule into the required position if necessary (Fig. 1).
4. Dispense a small amount of paste onto a mixing block and discard.
5. If the clinical situation requires it, place a retraction cord in the sulcus.
6. Rinse the sulcus with water and dry.
7. Place the material slowly in the sulcus around the prepared tooth until the sulcus is completely filled. Where necessary, use compression caps or other compression products.
8. Leave the material in place for max. 02:00 minutes, to enable the gingival retraction. While it is taking effect, keep the treated area dry.
9. Caution: Leaving the material in the mouth for too long can lead to irritation of the gingiva. Do not exceed the intraoral setting time.
10. Remove the material completely from the sulcus and mouth with a mixture of air and water and using suction.
11. If a stronger retraction is required, repeat the process from step 6. If necessary, a thin retraction cord can be used in addition.
12. Take the impression.

Interactions

Residues of the material can lead to inaccuracies or delays in the setting of the impression material.

Residual risks / side effects

There are no known side effects to date. The residual risk of hypersensitivity to components of the material cannot be ruled out.

Warnings / precautions

- For dental use only!
- Keep out of the reach of children!
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Do not ingest the material. In the event of unintentional ingestion, consult a doctor
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Performance characteristics of the device

Widens the sulcus.

Storage / disposal

- Store at 15 – 25 °C/59 – 77 °F.
- Do not use after the expiration date.
- Use the compule immediately after opening the aluminum bag. Throw away residual material in the compule.
- Disposal must comply with national regulations.

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