Silagum-Comfort

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English

Product Description

Permanently soft (non-hardening) relining material on the basis of A-silicones.

Indications

- Relining material for total, partial and implant-supported dentures

Basic Safety Information

- ▶ For dental use only!
- ▶ Keep out of reach of children!

Interactions

- ▶ Avoid having the relining material come into contact with latex gloves, since that can impede the setting process. If you have recently been in contact with latex gloves, thoroughly wash hands before working with the relining material. The use of PVC or PE gloves is recommended.

Handling the Automix Cartridge

Inserting the cartridge

1. Push up the lever [Fig. 1, Pos. A] on the back side of the Automix-Dispenser and pull the slider all the way out [Fig. 1, Pos. B].
2. Flip the plastic latch [Fig. 1, Pos. C] upwards, insert the cartridge and lock it in place with the plastic latch [Fig. 1, Pos. C].

Note: Make sure that the notches on the cartridge and the Automix-Dispenser match.

Attaching the Mixing-Tip

1. Remove the cartridge cap [Fig. 1, Pos. D] or used Mixing-Tip after turning it 90° counterclockwise, and then discard it.
2. Attach the new Mixing-Tip [Fig. 1, Pos. E].

Note: For optimal mixing results, DMG recommends using the mixing tips available from DMG. All mixing tips are also available in refill packs..

Note: Make sure that the notches on the tip and the cartridge match.

3. Turn the Mixing-Tip 90° clockwise to lock it into place.

Applying the Material

- ▶ When using a cartridge for the first time, express a pea-sized amount of material from the cartridge and discard it.
- ▶ Press the lever [Fig. 1, Pos. F] on the Automix-Dispenser to mix the material in the mixing tip and then apply it immediately.

Note: After completing the application, leave the used mixing tip on the cartridge; it serves as a cap for the cartridge!

Removing the cartridge

1. Push up the lever [Fig. 1, Pos. A] on the back side of the Automix-Dispenser and pull the slider all the way out [Fig. 1, Pos. B].
2. Flip the plastic latch [Fig. 1, Pos. C] upwards and remove the cartridge.

Information on Use

- ▶ The various times are listed in the “Specifications” section.
- ▶ Do not expose the relining to mechanical stress for at least 24 hours (e. g. cleaning with a brush).

Recommended Use

Direct relining in the patient's mouth

1. If necessary, remove old permanently soft relining material.
2. Trim off a minimum of 1 mm of all the areas needing to be relined. DMG recommends a rounded preparation of the edges of the dentures.

Note: Following this preparation, protect the denture surface from moisture, saliva, and monomer-containing resins.

3. Apply an even layer of primer (Silagum-Comfort Primer) in accordance with the manufacturer's information and let the solvent evaporate completely (approx. 1 minute). Be sure to completely cover the ridges and edge areas. Apply at least two coats.
4. Apply Silagum-Comfort first on the ridges and edges of the denture and then on the base of the denture. The layers should be at least 2 mm thick.
5. Remove saliva with a suction instrument.

Caution! Uncured silicone can trigger the patient's gag reflex.

► Therefore, to prevent the silicone from entering the patient's throat and esophagus, be sure that the patient is sitting upright and slightly bent forward when performing any functional movements.

6. Insert the denture into the patient's mouth within the specified application time and instruct the patient to bite into normal occlusion. Have the patient perform gentle swallowing and chewing movements (without exerting excessive pressure) for 5 minutes with his or her mouth closed.
7. Remove the denture from the mouth after approximately 6 minutes and let it harden.
8. Clean and disinfect the denture.

Note: If needed, carefully align the dry and grease-free relining by applying and modeling additional Silagum-Comfort. Let it set in the patient's mouth for 7 minutes or in a pressure pot (40 – 50 °C) for 5 minutes.

9. Remove excess material with sharp scissors or a scalpel. Smooth the trimmed edges with a suitable silicone trimmer.
10. Using a brush, apply the Silagum-Comfort Varnish to the treated areas in accordance with the manufacturer's instructions, avoiding air bubbles as much as possible, and then let dry.

Indirect relining at the laboratory

1. Following a functional impression, the gypsum is poured into the denture on the jaw side and mounted on a relining jig.

Note: Do not use polyether as the impression material. The impression material used should have a long application time and appropriate viscosity. DMG recommends applying a suitable separating medium (gypsum against resin).

2. Produce a negative by pouring the occlusal part of the denture or rows of teeth, which must be firmly fixed in its vertical relationship to the model. For this purpose, DMG recommends using a special relining device that facilitates the exact repositioning of model and negative (cuvette).
3. Remove impression material.
4. Trim the denture surface on the jaw side at least 1 mm and clean it.

Note: Following this preparation, protect the denture surface from moisture and monomer-containing resins.

5. Apply an even layer of primer (Silagum-Comfort Primer) in accordance with the manufacturer's information and let the solvent evaporate completely (approx. 1 minute). Be sure to completely cover the ridges and edge areas. Apply at least two coats.
6. Apply Silagum-Comfort first on the ridges and edges of the denture and then on the base of the denture. The layers should be at least 2 mm thick.
7. Reposition the denture on the model, and use the cuvette to completely envelop the negative.
8. Let the material set and harden for 10 minutes in the pressure pot at 40 – 50 °C.

Note: If needed, carefully align the dry and grease-free relining by applying and modeling additional Silagum-Comfort. Let it set in the pressure pot for 5 minutes (40 – 50 °C).

9. Remove excess material with sharp scissors or a scalpel. Smooth the trimmed edges with a suitable silicone trimmer.
10. Using a brush, apply the Silagum-Comfort Varnish to the treated areas in accordance with the manufacturer's instructions, avoiding air bubbles as much as possible, and then let dry.

Technical Data

	Silagum-Comfort	Silagum-Comfort Primer	Silagum-Comfort Varnish
Application time*	1:45 min.	-	2 min.
Setting time in mouth	6 min.	-	-
Setting (curing) time*	15 min.	-	15 min.
Evaporation time	-	1 min.	-

*The times listed here refer to normal room temperature 23 °C (73 °F) and normal relative humidity (50%) conditions. Higher temperatures reduce the times, lower temperatures increase them.

Disinfection of the Automix-Dispenser

The Automix-Dispenser and the trimmer and brush holders can be reused after disinfection and sterilization, but should be replaced if there are visible signs of wear (e.g. scratches).

The device can be disinfected and sterilized using a commercially available immersion disinfection solution. DMG recommends using only RKI (Robert Koch Institute) listed disinfection solutions. The Automix-Dispenser is also autoclavable

Storage and Shelf Life

- Store in a dry space at normal room temperature (15 – 25 °C / 59 – 77 °F).
- Do not store in the refrigerator or freezer.
- Keep out of direct sunlight.
- Leave used mixing tips on the cartridge after use. They serve as a cap.
- Do not use past the expiration date.

Composition

Addition-cured vinyl polysiloxanes, hydrogen polysiloxanes, fillers, pigments, additives, platinum catalyst

Packaging

Introductory Package	
1 cartridge containing 25 ml paste, 5 Automix-Tips, 1 Automix-Dispenser, 1 bottle primer containing 5 ml, 2 bottles varnish, 10 ml each, 25 brushes, 5 trimmers, 1 holder	REF 909087

Refill Pack	
1 cartridge containing 25 ml paste, 5 Automix-Tips	REF 909088
1 cartridge containing 50 ml paste, 12 Automix-Tips	REF 909951

Value Pack	
8 cartridges, each containing 50 ml paste	REF 909950

Accessories	
2 bottles Silagum-Comfort Varnish, each containing 10 ml	REF 909089
1 bottle Silagum-Comfort Primer, containing 5ml	REF 909090
1 Automix-Dispenser type 25 1:1	REF 110253
1 Automix-Dispenser type 50 1:1	REF 909507
50 Automix-Tips, yellow 1:1	REF 909200

Federal law (USA) restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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