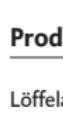
DMG Tray Adhesive

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English

Product description

Tray adhesive for bonding VPS materials with metal and plastic trays.

Intended purpose

For the bonding of addition-curing silicone impression materials (PVS) with minimal or non-retentive impression trays (metal, resin)

Limitations of use

There are no known limitations of use.

Indication

Dental situations requiring final impressions e.g. tooth loss, caries and trauma

Contraindication

Do not use the material where there are known allergies to any of the components or contact allergies.

Patient target group

Persons receiving treatment as part of a dental procedure.

Intended user

Dentist, dental assistant staff

Notes for use

- When using plastic impression trays, check the compatibility of the plastic with the tray adhesive.
- Ensure that the impression tray is clean, free of greasy residues and dry.

Recommended use

- Using a brush, apply a thin, even layer of the tray adhesive to the inside of the impression tray.
- Allow the tray adhesive to act for 3 minutes.
- Add the impression material to the tray within 30 minutes.

Interactions

When using a plastic impression tray, first check its compatibility with the tray adhesive.

Residual risks/side effects

There are no known side effects to date. The residual risk of hypersensitivity to components of the material cannot be ruled out.

Warnings/precautions

- Keep out of the reach of children!
- Avoid contact with the skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid contact with eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.

- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Product performance characteristics

Adhesion to metal ≥ 0.3 MPa

Storage/Disposal

- Store in a dry place at 2 to 25 °C (36 to 77 °F)!
- Close bottles immediately after use to prevent contamination as well as evaporation of the solvent.
- Do not use after the expiration date.
- Disposal must comply with national regulations.

Composition

Isoalkane, siloxane copolymer, modified resin, additives. Contains hexane.

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