# DeltaFil

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## Instructions for use English

### Product description

DeltaFil is a classic, radiopaque glass ionomer cement for restorative treatment. DeltaFil bonds chemically to enamel and dentin and releases fluoride ions. It is available in the shades White and Dark as well as A1, A2, A3 and A4 of the Vita tooth shade guide.

### Intended purpose

- Class I and II permanent restorations: areas not subject to occlusal load; also for sandwich technique
- Class I, II\*, and V permanent restorations; also for sandwich technique
- Class I, II, III, and V temporary restorations
- Extended fissure sealing
- Base/liner
- Class I, II, III, and V deciduous teeth restorations.

\* Also for areas subject to occlusal load, provided the isthmus measures less than half of the intercuspal area

### Limitations of use

Application is contraindicated if a relative dry isolation or the recommended application technique is not possible.

### Indication

- Loss of dental hard tissue due to caries, fractures, tooth abrasion or developmental disorders
- Secondary prevention of caries

### Contraindication

- Do not use directly on the exposed pulp.
- Do not use the material if there are known allergies to any of the components or contact allergies.

### Patient target group

People who are receiving dental treatment.

### Intended user

Dentist

### Notes for use

- Rinse instruments in water immediately after use.
- Observe the manufacturer's instructions for all materials that are used with the product.
- In deep cavities in areas close to pulp, provide protection in the form of a suitable liner, e.g., apply a small quantity of a strongly adhesive calcium hydroxide preparation.
- The time between mixing and applying should be kept to a minimum to avoid premature curing of the material.
- The formation of air bubbles should be prevented during application.
- Do not use DeltaFil without DeltaFil Conditioner.
- Once the DeltaFil restoration has been placed and finished, the use of DeltaFil Coat is optional.

### Using the Handmix version

#### Powder and liquid dosing

1. The glass ionomer cement is mixed at room temperature. The powder/liquid ratio is 4.96 g/1 g. This corresponds to 1 level scoop of powder and 1 drop of liquid.
2. Lightly mix up the powder in the bottle with the dosing spoon. Do not shake or turn the bottle over.
3. Remove the powder with the dosing spoon and level the dosing spoon on the scraper. Do not compress the powder.
4. Dispense a sufficient quantity of powder onto a clean mixing pad.
5. Hold the bottle with the liquid vertically, press lightly and dose the appropriate number of drops next to the powder. The drops should not contain any air bubbles.
6. After use, close the powder and liquid bottles carefully. There should be no powder residue on the scraper. Residues of liquid on the liquid bottle should be removed.

#### Mixing

Using a plastic spatula, divide the powder into two equal parts, and mix the first portion with all the liquid for 15 s. Add the remaining powder and mix for another 15 s until a homogeneous consistency is achieved.

#### Recommended use

1. Prepare the tooth as usual. When shaping the cavity it should be ensured that the material layer is not thinner than 0.5 mm at any point.
2. Then rinse the cavity thoroughly with water, remove excess water (e.g., with a cotton pellet or blow dry lightly with a gentle air flow). The prepared surface should still be slightly damp. The dentin must not become too dry.
3. Protect areas near the pulp.
4. Using DeltaFil Conditioner (observe the relevant instructions for use!): apply DeltaFil Conditioner with a cotton pellet or mini applicator sponge to the prepared area and leave in place for 10 s. Then rinse thoroughly with water. Remove excess water as described above. The surface should still look moist for the next steps.

**Note:** DeltaFil Conditioner is an aqueous polyacrylic acid solution for treating the smear layer and conditioning the enamel and dentin, optimizing the bond. Apply as described above. Carefully reseal the bottle of the DeltaFil Conditioner after use.

5. If necessary, place a matrix.
6. Apply the mixed material into the cavity, compress it, adapt it to the edges of the cavity and model the surface. DeltaFil can be molded for up to 120 s from the start of mixing.
7. Immediately after modeling, care should be taken to prevent moisture contamination and excessive drying. In the event that this cannot be guaranteed, apply DeltaFil Coat immediately and light-cure it. Follow the relevant instructions for use!
8. Then let the material cure for approx. 260 s.
9. The restoration can be finished as usual, e.g., with abrasive tooth grinders and finishing burs, 360 s after mixing has been started. Grinding can be performed with or without water spray.
10. DeltaFil Coat can be optionally applied and cured to protect the restoration. Follow the relevant instructions for use!

#### Timing

Mixing time	≥ 30 s
Working time (23 °C/73 °F) from the start of mixing*	≤ 120 s
Net setting time (37 °C/99 °F)	≥ 260 s
Begin the final polishing from the start of mixing	≥ 360 s

\* The specified times are valid at a room temperature of 23 °C (73 °F) and a normal relative humidity of 50 %. Higher temperatures shorten these times; lower temperatures lengthen these times.

#### Interactions

No interactions are currently known.

#### Residual risks/side effects

- Please check the product for damage and visible contamination before use.
- Hypersensitivity and/or allergic reactions cannot be ruled out in individuals who are predisposed.
- Improper use of the product can lead to excessive loss of dental hard tissue, eye irritation, respiratory tract irritation, skin or mucous membrane irritation and/or damage, choking, and infection.
- A low residual risk of unknown side effects cannot be ruled out.

## Warnings/precautions

- For dental use only!
- Avoid contact with the skin! In the event of accidental contact with the skin, immediately wash the affected area thoroughly with soap and water.
- Wear safety goggles and avoid contact with eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Observe the safety data sheet.
- Follow the manufacturer's instructions for other products used with the material/materials.
- The manufacturer (productsafety@mtcompanies.com) and the responsible registration authorities must be notified of any serious incidents involving this product.

## Product performance characteristics

Complies with ISO 9917-1:2025 chemical type C and application type C

## Storage/disposal

- Store dry, away from light and tightly sealed at 15 – 25 °C (59 – 77 °F).
- Do not use after the expiration date!
- Disposal must comply with national regulations.

## Composition

Ionomer glass, polyacrylic acid, water, tartaric acid, additives.

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