





Stays where you need it – flowable when you want it to be

Honigum Pro makes all impression tasks in the dental field easier and was specifically developed for implant prosthetics.

The unique precision VPS offers you impressive advantages:


- High precision and clean handling
- No dripping during application, no undesirable run-off
- Excellent flowability under pressure
- Effective removal due to extraordinary toughness
- Exact impressions
- Long working time with short setting time in the mouth thanks to "Snap-Set" technology

Benefit from Honigum Pro's special features in your everyday work. The DMG patented material structure ensures precise results quickly, safely and cleanly and achieves top ratings in everyday tests and studies – with patients and dentists alike.

Thanks to its excellent stability when no pressure is applied, Honigum Pro can be safely inserted and remains in the applied site. Under pressure – when the tray is inserted, for example – the material reliably reaches the smallest spaces even in a moist environment, ensuring accurate impressions. You decide when Honigum Pro stays steady and when it flows.




1.1 Applying the Light material.

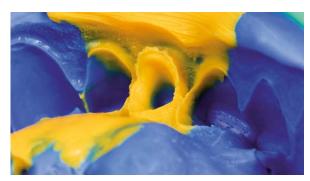

1.2 The homogeneous impression as a result.



2.1 Insertion of the impression tray filled with Honigum Pro Putty and Honigum Pro Light.

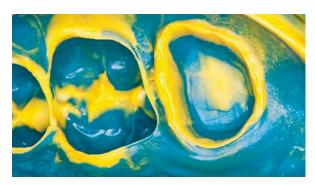

2.2 Detailed impression taking with Honigum Pro Putty and Honigum Pro Light.


Honigum Pro - Clear benefits in daily routine.


Impression materials need to produce satisfactory results, even when used in moist conditions.

Honigum Pro's special crystal structure – the rheologically active matrix – offers the decisive added value for both you and your patients.

In tests and comparisons with competitor products, Honigum Pro demonstrates the best flow behaviour **under moist conditions** in the artificial sulcus in the area of the subgingival preparation margin. The optimum interaction of good hydrophilicity and unique rheological properties guarantees you absolute accuracy when taking impressions.

But Honigum Pro offers you another significant advantage in your everyday work. The **excellent combination of tear resistance and elongation resistance** – scientifically referred to as toughness – is crucial for safe removal from the mouth. This is another factor that enables Honigum Pro to set itself apart from other products, ensuring the highest restoration quality and saving time and money in equal measure. After all, an impression that tears when it is removed from the mouth is a wasted impression.

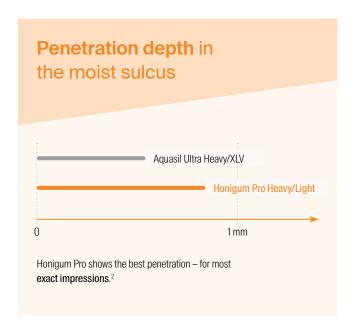

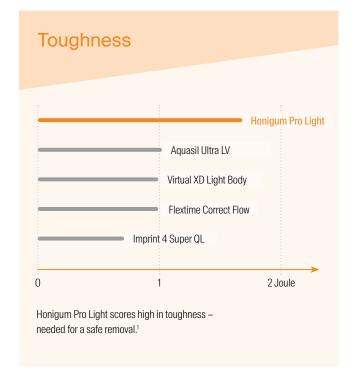


¹ Source: Internal DMG measurements, data on file, 2011.


Source: Internal Bivid measurements, data of nie, 2011.
 Source: Dr B. Wöstmann, Justus-Liebig-Universität Gießen, Poliklinik für Zahnärztliche Prothetik.

"Snap-Set" technology for maximum comfort

Generous working time with shorter setting time in the mouth – DMG's "Snap-Set" technology takes into account the needs of the dentist and the patient alike.

- Honigum Pro shortens the setting time in the mouth and thus ensures significantly more comfort during treatment.
- This minimises the stress for both the patient and the dentist providing treatment.
- The dentist can work in a relaxed and safe manner – even intraorally – because the viscosity remains constant over a generous period of time.
- Distortions caused by material that has already set are avoided.
- Faster cross-linking times reduce inaccuracies caused by micromovements.

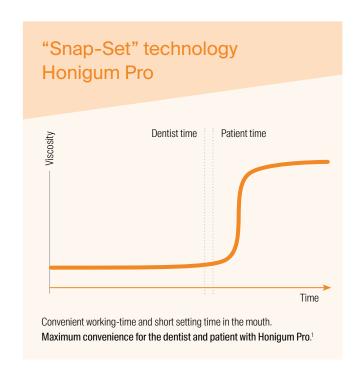

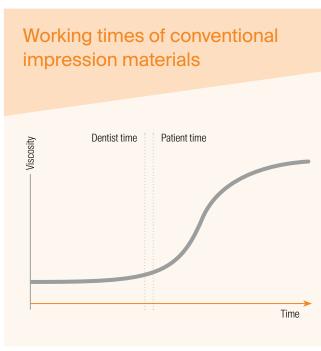

Easier, faster and safer:

The MixStar eMotion

Metering, mixing and application of Honigum Promade easy.

- Easier: With just one touch of a button and the two timers for working time and setting time in the mouth, you can operate the MixStar eMotion intuitively yet in line with individual requirements.
- Faster: Fill syringes or impression trays in three speed levels and save up to 20 seconds compared to the previous model.
- Safer: Electronic control ensures exact metering and minimal material waste. Errors are avoided by storing the correct times for working and setting in the mouth.



The MixStar eMotion offers you electrical perfection when working Honigum Pro.

DMG Retraction Paste – your straightforward impression-taking assistant



DMG Retraction Paste is the ideal complement for accurate impressions and restorations with Honigum Pro.

- An individually bendable compule allows for extremely easy application at any point in the patient's mouth.
- The material remains at the applied site, optimally takes effect there and is clearly visible in the sulcus.
- **The homogeneous consistency ensures optimum** results for temporary sulcus retraction.

Excellent results in implant prosthetics

For implant impressions, precise transfer of the oral situation to the working model is crucial for creating the accurate-fit superstructure. Subsequent accuracy of fit in the patient's mouth is only guaranteed if the position of the impression posts is accurately recorded and reproduced. With its unique properties, Honigum Pro also ensures excellent results in implant prosthetics.

- With its excellent flowability, Honigum Pro encloses the impression posts or transfer coping particularly well and fixes them reliably. Torsions are prevented by the fast setting behaviour and the high final hardness – especially with Honigum Pro Heavy.
- Due to the optimum stability, the material remains at the applied site and reproduces gingival structures in detail – your guarantee for harmonious integration of implant prosthetics into the gingival contours.



Our tip:

Honigum Pro is available in different variants that can be combined exactly according to your requirements. The Fast variants of Honigum Pro shorten treatment times even further, while a mixture of Honigum Pro Heavy and Honigum Pro Light in the double-mix technique is recommended for combined work involving implants and prepared teeth. With a pure implant impression Honigum Pro Heavy is also suitable as a monophase impression material.

An overview of the different Honigum Pro variants can be found on the next page.

The **right Honigum** Pro for every impression task

		Honigum Pro Light	Honigum Pro Mono	Honigum Pro Heavy	Honigum Pro Putty
Crown and bridge impressions		•	•	•	•
Inlay and onlay impressions		•	•	•	•
Implant impressions				•	
Fixation impressions				•	
Functional impressions			•		
Pressure-spot indication			•		
Viscosity		Low viscosity	Medium viscosity	High viscosity	High viscosity
Setting behavior	Normal-curing	•	•	•	•
	Fast curing	•	•	•	
Technique	Monophase		•		
	Double-mixing	•		•	•
	Correction	•		•	•
Syringe material		•	_		
Insertion resistance	Low	•	_		
	Medium		•	•	

Technical data

	Honigu Lig				Honigum Pro Heavy			Honigum Pro Putty						
Honigum Pro		(1)		(1	(1)	(1)	((1)	(1)				
		Fast		Fast		Fast		Fast		Fast		Soft	Soft Fast	Rigid Fast
Contents [ml]	50	50	380	380	50	50	380	380	50	50	380	450×2	450×2	450×2
Mixing time [min]												0:30	0:30	0:30
Intraoral working time [min]	≤ 1:00	≤ 0:30	≤1:00	≤ 0:30	≤1:00	≤ 0:30								
Max. extraoral working time [min]	≤ 2:45	≤ 1:30	≤ 2:45	≤1:30	≤ 2:45	≤ 1:30	≤ 2:45	≤ 1:30	≤ 2:45	≤ 1:30	≤ 1:45	≤ 3:15	≤ 1:45	≤ 1:45
Setting time in the mouth [min]	≥ 3:15	≥ 2:00	≥ 3:15	≥2:00	≥ 3:15	≥2:00	≥ 3:15	≥2:00	≥ 3:15	≥ 2:00	≥ 3:15	≥ 3:15	≥ 1:45	≥ 1:45
Dimensional change after 24 hrs [%]	≤ 0.35	≤ 0.35	≤ 0.4	≤ 0.4	≤ 0.4	≤ 0.4	≤ 0.4	≤ 0.4	≤ 0.4	≤ 0.4	≤ 0.5	≤ 0.5	≤ 0.5	≤ 0.5
Compression set [%]	≤ 0.35	≤ 0.35	≤ 0.4	≤ 0.4	≤ 0.4	≤ 0.5	≤ 0.5	≤ 0.5	≤ 0.5	≤ 0.6	≤ 0.7	≤ 0.5	≤ 0.5	≤ 0.5
Shore-A-hardness (24h)	≈55	≈55	≈61	≈ 61	≈61	≈ 61	≈76	≈74	≈74	≈74	≈73	≈68	≈68	≈75
Strain in compression [%]	≈4.0	≈ 4.0	≈2.8	≈2.8	≈2.8	≈2.8	≈1.8	≈1.8	≈2.0	≈2.0	≈2.0	≈2.2	≈2.2	≈ 1.7
Shade	•	•	•	•	•	•	•	•	•	•	•	•	•	•

Clear benefits for your everyday work

Honigum Pro Light

High-precision, VPS-based correction impression material with a light consistency

Indications:

Crown and bridge impressions, inlay and onlay impressions

Benefits for the dentist:

- Intelligent flow properties and extraordinary stability
- Excellent hydrophilic properties
- Registers even the finest details
- Very high toughness
- Easily readable shade contrast

Honigum Pro Heavy

Precision, VPS-based impression material with a heavy consistency

Indications:

Crown and bridge impressions, inlay and onlay impressions, all types of pickup impressions (e.g. for implants)

Benefits for the dentist:

- High final hardness
- Convenient insertion resistance
- Trouble-free and safe removal
- Consistently homogeneous, bubble-free material quality

Honigum Pro Putty

Precision, VPS-based impression material with a putty-like consistency

Indications:

Crown and bridge impressions, inlay and onlay impressions

Specialities:

Real putty consistency from the DMG MixStar cartridge

Benefits for the dentist:

- Outstanding recovery after deformation
- Excellent dimensional stability
- Can be shaped by hand

Honigum Pro Mono

Monophase VPS-based impression material with medium consistency

Indications:

Crown and bridge impressions, inlay, onlay and functional impressions, as well as pressure-spot indications and all types of pickup impressions (e.g. for implants)

Benefits for the dentist:

- One-step precision
- Trouble-free and safe removal
- Highly hydrophilic and superior flow behaviour
- Precise reproduction of the preparation margin
- Very high tear resistance



For more information, please visit our website: www.dmg-dental.com

Packaging

Honigum Pro Light Economy Pack: Packaging: 2 Cartridges @ 50 ml 10 Automix Tips Yellow 1:1 8 Cartridges @ 50 ml 40 Automix Tips Yellow 1:1 10 Intraoral Tips Yellow 40 Intraoral Tips Yellow Honigum Pro Light REF 929831 Honigum Pro Light REF 929835 Honigum Pro Light Fast REF 929832 Honigum Pro Light Fast REF 929836 **Honigum** Pro Mono Packaging: **Economy Pack:** 2 Cartridges @ 50 ml 4 Automix Tips Green 1:1 8 Cartridges @ 50 ml 16 Automix Tips Green 1:1 4 Intraoral Tips Yellow 16 Intraoral Tips Yellow Honigum Pro Mono REF 929830 Honigum Pro Mono REF 929834 Honigum Pro Mono Fast REF 929976 Honigum Pro Mono Fast REF 929977 5 Cartridges @ 380 ml 50 MixStar Tips Blue 1 Cartridge @ 380 ml 10 MixStar Tips Blue Honigum Pro Mono REF 929568 Honigum Pro Mono REF 929569 Honigum Pro Mono Fast REF 929978 Honigum Pro Mono Fast REF 929979 **Honigum** Pro Heavy **Economy Pack:** Packaging: 2 Cartridges @ 50 ml 4 Automix Tips Green 1:1 8 Cartridges @ 50 ml 16 Automix Tips Green 1:1 Honigum Pro Heavy REF 929766 Honigum Pro Heavy REF 929837 Honigum Pro Heavy Fast REF 929838 Honigum Pro Heavy Fast REF 929839 1 Cartridge @ 380 ml 10 MixStar Tips Blue 5 Cartridges @ 380 ml 50 MixStar Tips Blue Honigum Pro Heavy REF 929537 Honigum Pro Heavy REF 929538 Honigum Pro Heavy Fast REF 929638 Honigum Pro Heavy Fast REF 929637 **Honigum** Pro Putty Packaging: **Economy Pack:** 1 Cartridge @ 380 ml 10 MixStar Tips Blue 5 Cartridges @ 380 ml 50 MixStar Tips Blue Honigum Pro Putty REF 929374 **Honigum Pro Putty** REF 929375 2 Jars @ 450 ml 8 Jars @ 450 ml REF 929364 REF 929365 Honigum Pro Putty Soft Honigum Pro Putty Soft Fast Honigum Pro Putty Rigid Fast Honigum Pro Putty Soft Honigum Pro Putty Soft Fast Honigum Pro Putty Rigid Fast REF 929369 REF 929370 REF 929371 REF 929366 Honigum Pro Scan Packaging: Packaging: 2 Cartridges @ 50 ml 4 Automix Tips Green 1:1 1 Cartridge @ 380 ml 10 MixStar Tips Blue Honigum Pro Scan Heavy Honigum Pro Scan Heavy REF 929981 REF 929983 2 Cartridges @ 50 ml 10 Automix Tips Yellow 1:1 10 Intraoral Tips Yellow Honigum Pro Scan Light REF 929984 **DMG Retraction Paste** Accessories: 1 Automix Dispenser Type 50 1:1 50 Automix Tips Yellow 1:1 REF 909507 REF 909200 REF 909201 Packaging: 25 Compules @ 0.37 g REF 919381 50 Intraoral Tips Yellow 50 Automix Tips Green 1:1 REF 909490 Accessories: 50 Intraoral Tips Transparent REF 909400 REF 213122 Safetip Dispenser 1 MixStar eMotion 230 V Int. 50 MixStar Tips Blue 1 Safetip Dispenser Pro REF 900480 REF 903334 1 Bottle @ 10 ml DMG Tray Adhesive REF 909394

Honigum Pro For impression masterpieces















DMG

Chemisch-Pharmazeutische Fabrik GmbH

Elbgaustraße 248 22547 Hamburg Germany
Fon: +49. (0) 40. 84 006-0 Fax: +49. (0) 40. 84 006-222
info@dmg-dental.com www.dmg-dental.com
www.facebook.com/dmgdental

