



**PRODUCT OVERVIEW**

**LuxaPick-up**

**Packaging:** 1 Cartridge 76 g, 15 Automix-Tips, Shade Pink REF 214007

**Luxatemp-Glaze & Bond**

**Packaging:** 1 Bottle @ 5 ml, 25 brushes REF 211927

**Accessories:**  
 1 Automix-Dispenser Type 50 10:1 REF 110411  
 1 Automix-Plunger 50 10:1 REF 110412  
 45 Automix-Tips Blue 10:1 REF 110409



# LuxaPick-up

The professional prosthesis that can do more.



The professional  
prosthesis  
that can do more.  
**LuxaPick-up.**

## Easy fixation and precise relining of dentures.

When it comes to perfectly fitting dentures, LuxaPick-up makes the work much easier. The gingiva-shaded composite based on multi-functional methacrylates is your best choice for:

### Direct hard relining as compensation for changes in the mucosal bed

### Secure hold through utilization of stabilizing retention elements (pick-up technique)


The retention elements can be directly polymerized intraorally. This ensures a tension-free, secure hold on implants. The removable dentures can be easily adapted to changing mucosal beds – thanks to direct chairside use. LuxaPick-up offers a quick and cost-effective way to restore functionality. No time delay due to external laboratory work, no inaccuracies.

### GOOD REASONS:


- Direct hard relining of dentures
- Fixation of secondary parts and connection elements in implant-borne dentures
- Particularly bio-compatible: no MMA, no peroxide
- Tension-free, reliable support
- Direct chairside use
- Ideal for repairing prostheses and individualization of intermediate dentures




Prepare a circular undercut area in the base of the prosthesis.

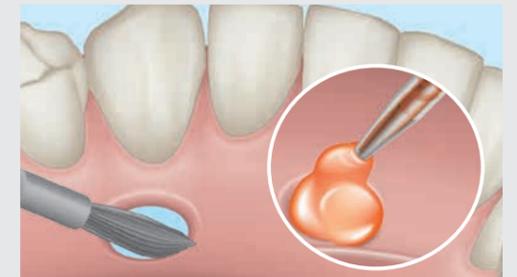

Apply Luxatemp-Glaze & Bond and light cure. Then apply LuxaPick-up.

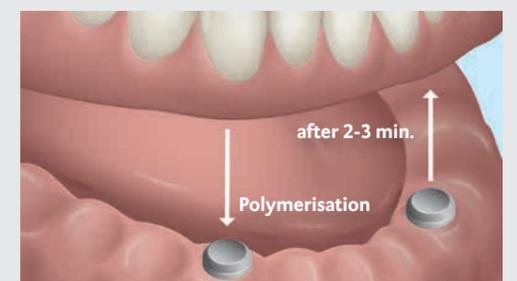

Insert and secure the prosthesis with moderate pressure.

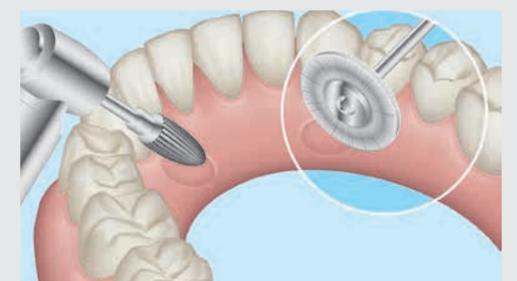

Grind off excess and finally polish the ground areas.