DentaMile Cure MC

User Manual

Inhalt / Contents

User Manual – English	1
Benutzeranleitung – Deutsch	4
Manual do usuário – Português	7
用户手册 – 中文版	10
Instrukcja obsługi – Polski	12
Gebruikshandleiding – Nederlands	15

User Manual – English

Safety rules

Warning: this UV polymerizer uses ultra high-energy UV LED sources that can seriously damage health, especially in eyes and skin; therefore, avoid direct or indirect exposure to these emissions even for short periods of time.

Warning: even though the *DentaMile Cure MC* oven is equipped with an automatic switch off system of the UV LEDs in case of accidental opening of the door, avoid this practice during the execution of a drying program. If you need to open the door, pause the program or stop it before opening the device.

Warning: the safety systems on the *DentaMile Cure MC* are intended as emergency aids: not try tamper in any way the integrity of the UV oven or any part thereof, do not tamper with the closed-lid control system. In case of a malfunction of the system or in case of incorrect lighting of the LEDs with the door open, immediately remove the power plug from the outlet and contact service.

Warning: The *DentaMile Cure MC* is powered at 220V mains voltage, so avoid tampering with the equipment in all its parts to avoid the risk of electrocution. In case of malfunction of the device contact qualified assistance.

Warning: the *DentaMile Cure MC* has a touchscreen interface and does not use commands or mechanical buttons. For this reason, even when the oven is not in use, it remains powered from the network. Cut power to the oven each time it is not used for long periods, especially.

Warning: *DentaMile Cure MC* utilizes the heat generated by the LEDs to accelerate the curing process of the resins, The treated objects and the oven itself may be very hot.

Warning: *DentaMile Cure MC* uses the heat generated by LEDs and nitrogen gas for the resin curing process. Do not leave *DentaMile Cure MC* unattended to avoid the risk of fire.

Warning: the resins during polymerization can generate a lot of heat, especially if collected in large quantities. Never leave the *DentaMile Cure MC* unattended to avoid the risk of fire.

Warning: the resins placed for curing in *DentaMile Cure MC* could generate fumes from harmful volatile substances to health. Position and use *DentaMile Cure MC* in a well-ventilated place.

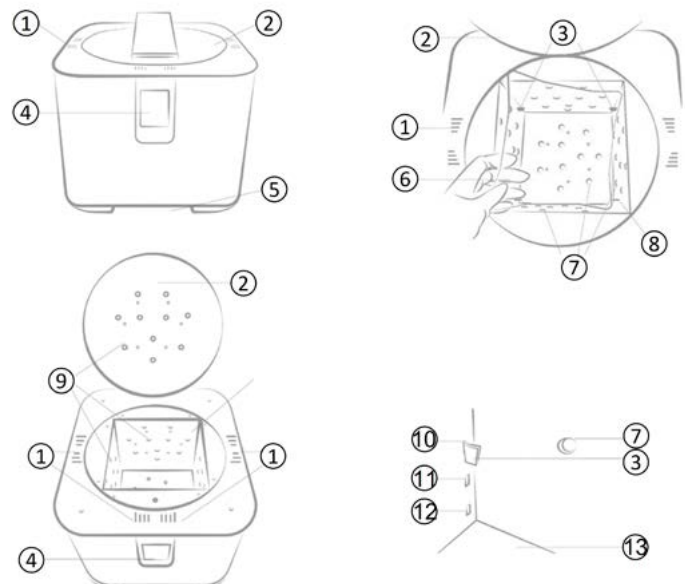
Warning: The fan located on the rear of the *DentaMile Cure MC* has the task of controlling the temperature of the device through numerous air vents placed in different points of the *DentaMile Cure MC*. Do not obstruct in any way the ducts, the openings to the outside and the fan itself, to avoid damaging the *DentaMile Cure MC* itself.

Warning: the *DentaMile Cure MC* is a device for professional use. Keep in a safe place and accessible only to qualified and aware of the rules described here. Do not allow children to use it.

Warning: Place *DentaMile Cure MC* away from heat sources, away from flammable materials and liquids that may cause risk to safety reason. Do not wet, wipe with a dry cloth.

General notes


Thank you for: choosing a DMG product. Each of our devices is made with great care and rigor, the continuous technological research and the goodness of the project make *DentaMile Cure MC* the indispensable tool for curing resin products.

1. Cooling ducts
2. Curing chamber lid
3. Tray supports
4. Touch screen
5. Fan output (not pictured)
6. Post curing tray
7. LED bulbs

8. Curing chamber
9. LED light source
10. Top hole
11. Middle hole
12. Bottom hole
13. Base of curing chamber

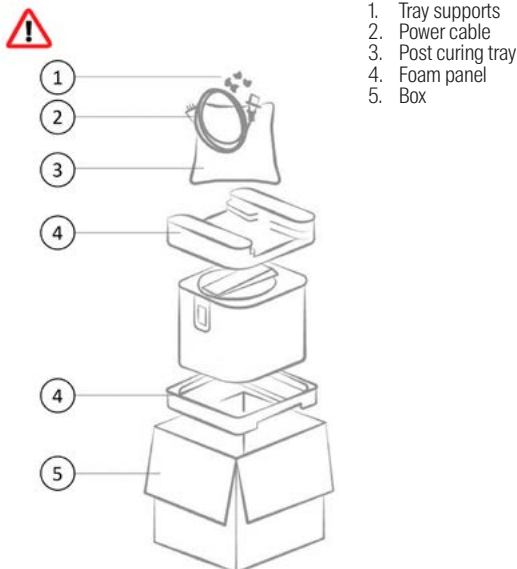First start

The *DentaMile Cure MC* system has a microprocessor-controlled touch-screen interface and intuitive software that allows you to control every aspect of the curing process of your resin objects.

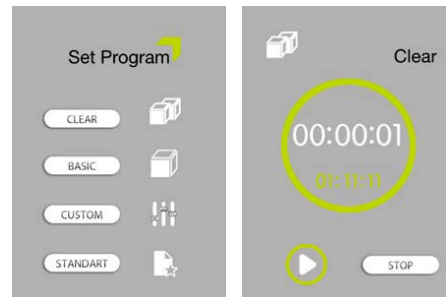
The *DentaMile Cure MC* system has three curing programs: two for standard use to finalize small and large products, and a third called Custom that allows you to create a customized program for curing your resins. All three programs use the controlled emission of several very powerful 405nm and 365nm UV LEDs and the accelerating effect of heat, thanks to the thermoregulation system. The multiple action of the many UV emitters allows the 360° irradiation of the piece without having to rotate it.

Set Program


From the power-on screen, it automatically switches to the program screen. In this condition the oven is powered and ready to start the program chosen by the user.

Clear Program


The Clear program is designed for curing objects produced with transparent and delicate resins. The UV emission is reduced to 10% and the exposure time is 20 minutes. The program starts by clicking on the CLEAR icon.

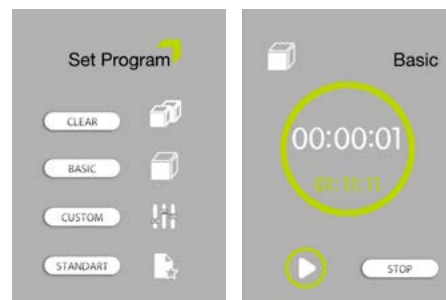Basic Program

The Basic program is designed for curing large objects that take up most of the curing volume. The UV emission is 100% of its power and the exposure time is 40 minutes. The temperature curve will reach 40° C during the program. The program starts by clicking on the Basic icon.

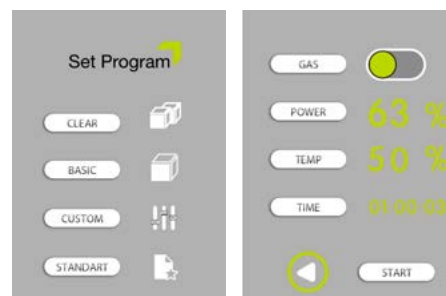Custom Program

The Custom program is designed for the polymerization of resins that need a customized process.

The Custom program allows you to adjust

- the UV power from 1% to 100%
- the exposure time from 1 second up to 4 hours
- the temperature that will be reached during the program, from 20°C to 60°C.

The program starts by clicking on the CUSTOM button. After selecting the program, you will be asked to enter the parameters for the polymerization. By clicking on the parameter button you have access to the set that will allow you to enter the required setting.

After entering the desired settings, click on the “back” symbol to return to the Set Parameter page; when all the parameters have been inserted, click on START to start the program.

During the program it is possible to interrupt it by clicking on STOP or put it in PAUSE.

During the program the display will go on stand-by by dimming. To access the display, just touch the screen.

Alarms

The product should never be opened while a curing program is running, unless the appropriate adapter is used.

DentaMile Cure MC however, has an automatic emergency shut-down system connected to the closing door. In case of accidental opening of the door the *DentaMile Cure MC* will stop working and an alarm screen similar to the one on the right will appear.

Warning! If the program does not stop immediately, turn off the power to the *DentaMile Cure MC* and contact support.

Maintenance

If the product does not turn on, check that the power cord is inserted correctly in the power socket and in the plug in the rear panel.

Environmental operating conditions

Keep the product in a ventilated place away from sources of heat, away from flammable materials and liquids that may cause a safety risk. Keep *DentaMile Cure MC* at least 20 cm away from the wall. Do not obstruct in any way the pipes, their openings to the outside and the fans on the rear panel and on the bottom to avoid damaging the *DentaMile Cure MC* itself. Keep *DentaMile Cure MC* in an environment with temperatures of 18° C to 28° C

WiFi connection

Click on the upper right corner of the display and follow the directions. Connect the PC to the WIFI of the product. The name of the network can be found on the display. Enter your password and access the Dashboard with your browser.


Technical features

External dimensions	320x330xH246
Curing chamber dimensions	145x145XH120
Emitter type	hi-power LED
Number of emitter	30
Irradiance system	OmniRay 360 °Irradiance
Wavelength	3
Emission centralization	λ1 365nm - λ2 385nm - λ3 405nm
Emission opening	+/- 20nm system: Eco-passive

Heating control

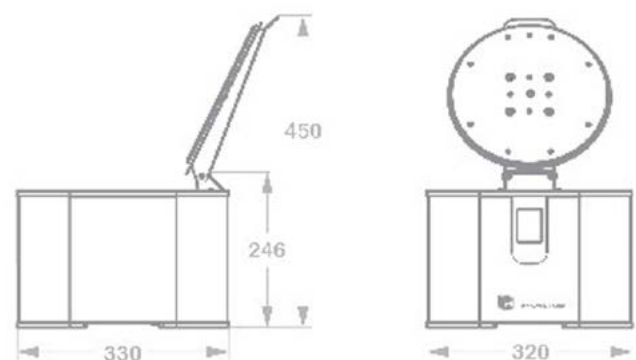Interface	2,4" TFT touch screen
Connectivity	2.4Ghz WiFi
Power supply	External
Power supply voltage	110-230V 6-4A 50-60Hz

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