DentaMile Wash MC

User Manual

Inhalt / Contents

| | |
|----------------------------------|----|
| User Manual – English | 1 |
| Benutzeranleitung – Deutsch | 3 |
| Manual do usuário – Português | 5 |
| 用户手册 – 中文版 | 7 |
| Instrukcja obsługi – Polski | 9 |
| Gebruikshandleiding – Nederlands | 11 |

User Manual – English

Safety rules

Warning: *DentaMile Wash MC* cleans the surface resin not cured by the 3D printer using chemicals like the IPA. *DentaMile Wash MC* is a professional product and must be used by qualified personnel, following instructions and warnings

Warning: The *DentaMile Wash MC* is powered at 220V mains voltage, so avoid tampering with the equipment in all its parts to avoid the risk of electrocution. In case of malfunction of the device contact qualified assistance.

Warning: We are not producers and suppliers of detergent products for resins. We recommend that you carefully read the instructions of the product used for cleaning resin objects.

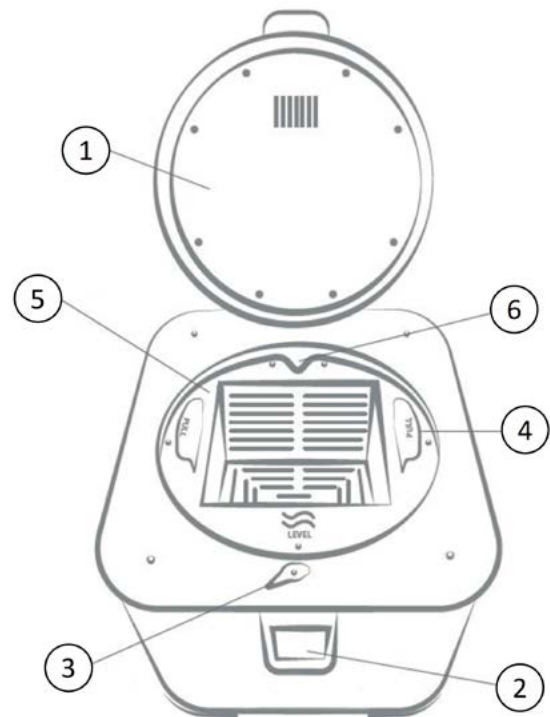
Warning: Use the *DentaMile Wash MC* in ventilated areas. Many detergents are flammable and highly volatile, after use place the detergent in the appropriate storage box. Do not use the *DentaMile Wash MC* near sources of heat or sparking.

Warning: After use, always empty the tank from the detergent product. Be careful when emptying the tank.

General notes

Thank you for choosing a DMG product. Each of our devices is made with great care and rigor, the continuous technological research and the goodness of the project make *DentaMile Wash MC* the indispensable tool for cleaning resin products.

DentaMile Wash MC – let's get to know it


- | | |
|-------------------------|-------------------------|
| 1. Compartment lid | 4. Tank's handles |
| 2. Display Touch screen | 5. Removable inner tank |
| 3. Lock | 6. Drainage spout |

First start

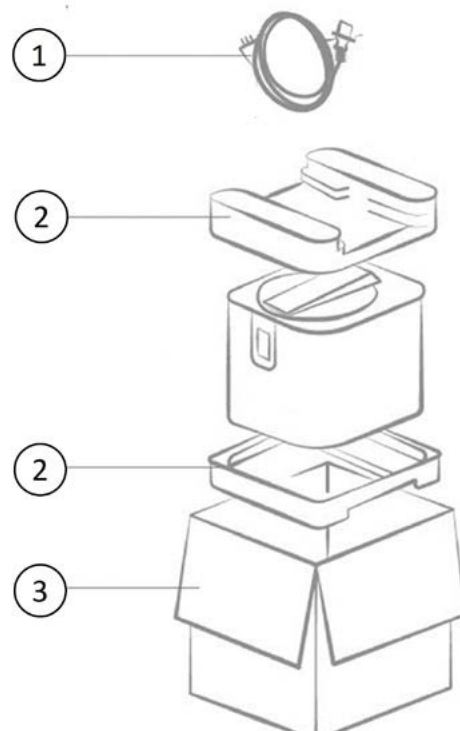1. Power cable
2. Foam panel
3. Box

Programs

The *DentaMile Wash MC* system has a microprocessor-controlled touch-screen interface and intuitive software that allows you to control every aspect of the cleaning process of your resin objects.

The programs

DentaMile Wash MC has 3 speed programs

Low: slow washing, dedicated to very delicate pieces

Med: standard washing

High: strong and incisive washing

By clicking on each program you can establish the washing time and start your process.

End of the program

At the end of the program the display will show a message. Remember to get out the pieces from the tank at the end of the wash. If the resin remains in contact for a long time with the product for cleaning it may be damaged (read the warnings of the product for cleaning). After having dried your resin parts, proceed with the curing with your *DentaMile Cure MC* by setting the required program.

Removal of the tank

To remove the tank, carry out the following operations: raise the handles with the word »PULL« until the end of the stroke. As soon as the handles reach the end of the stroke, lift the tank upwards always taking it from the handles. If you want to take the pieces you have deposited in the tank, raise the handles and you will see that the bottom grid will rise up to the surface of the liquid.

Tips

Remember to check the effectiveness of your cleaning fluid that runs out after some washes. Follow the instructions given with the cleaning product for measuring effectiveness.

Warning! If the program does not stop immediately, turn off the power to the *DentaMile Wash MC* and contact support.

Maintenance

If the *DentaMile Wash MC* does not turn on, check that the power cord is inserted correctly in the power socket and in the plug in the rear panel. After each use Thoroughly clean the tank with resin cleaning liquid. The residual resins left on the bottom could damage the rotation movement.


Environmental operating conditions

Keep *DentaMile Wash MC* in a ventilated place away from sources of heat, away from flammable materials and liquids that may cause a safety risk. Keep *DentaMile Wash MC* at least 20cm away from the wall. Do not obstruct in any way the pipes, their openings to the outside and the fans on the rear panel and on the bottom to avoid damaging the *DentaMile Wash MC* itself. Keep *DentaMile Wash MC* in an environment with the temperatures of 18° C to 28° C.

Features
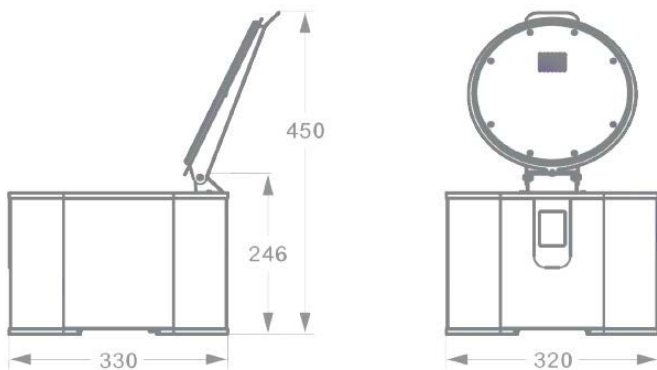
| | |
|-------------------------|----------------------------|
| External dimension | 320x330xh246 |
| Washing tank dimensions | 145x145xh105 |
| Washing system | Contact-Less agitator |
| Washing Mode | basket of platform |
| Liquid capacity | 3.5lt |
| Compatibility | universal adapter (option) |
| Programs | Strong – medium – delicate |
| Management | 16bit IDM processor |
| Interface | 2.4 „ TFT touch screen |
| Controls | auto off |
| Movement | stepper |
| Power supply | external |
| Power supply voltage | 100-230V 0.3A |

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