Luxatemp MaxProtect

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Dentalglas, EBPADMA, Urethanacrylat-Oligomer, Polyester, Urethanmethacrylat, SiO₂, TEDMA, HEMA, Additive.

Instructions for use

English

Product description

Luxatemp MaxProtect is a self-curing composite used for manufacturing long-term provisionals that are cemented with a permanent or temporary luting cement. The 2-component material is based on multi-functional methacrylates and is automatically mixed.

Intended purpose

Manufacture of long-term provisionals: crowns, bridges, inlays, onlays, partial crowns and veneers.

Limitations of use

Do not use the material for permanent restorations.

Indication

- Major dental defects
- Misalignment of the teeth
- Tooth loss

Contraindication

Do not use the material where there are known allergies to any of the components or contact allergies.

Patient target group

Persons receiving treatment as part of a dental procedure.

Intended user

Dentist, dental technician

Notes for use

- With the Automix version, the material is applied using a 10:1 dispenser.
- When using an Automix Cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- In order to achieve an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are also available in refill packs.

Using the Automix Cartridge (for image, see "Handling")

Use of the Automix Cartridge

1. Push up the lever [A] at the back of the Automix Dispenser and pull back the slide [B] all the way.
2. Lift up the plastic lever [C], insert the Automix Cartridge and lock it using the plastic lever [C].

Note: Make sure that the notches on the Automix Cartridge line up with those on the Automix Dispenser.

Attaching a mixing tip

1. Remove the cartridge cap [D] or used mixing tip after turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

Note: Make sure that the notches on the mixing tip line up with those on the Automix Cartridge.

3. Twist the mixing tip clockwise by 90° to lock it.

Applying the material

- When using an Automix Cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- Mix the material in the mixing tip using the lever [F] on the Automix Dispenser and apply directly.

Note: After the application, leave the used mixing tip on the Automix Cartridge as a seal!

Removal of the Automix Cartridge

1. Push up the lever [A] at the back of the Automix Dispenser and pull back the slide [B] all the way.
2. Lift up the plastic lever [C] and remove the Automix Cartridge.

Disinfection of the Automix Dispenser

- The Automix Dispenser can be reused after disinfection, although it should be replaced if there are visible signs of wear.
- Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch Institute).

Recommended use

1. Before preparing the crown or bridge or prior to extraction of a tooth, take an impression using alginate, silicone or a thermoplastic impression material. Alternatively, the temporary can be made using a vacuum-formed splint matrix prepared in the laboratory. Adhere precisely to the relevant manufacturer's instructions!

Note: In a silicone impression, undercuts should be compensated for and, if necessary, relief channels should be cut.

2. Dry the prepared teeth and compensate for any undercuts present in the preparation.
3. Lightly coat the prepared teeth, surrounding tissue and any present composite core build-ups with Vaseline or a similar separating agent.

Note: The convergence angle must be kept as small as possible in the preparation. When working on splinted crowns and bridges, make sure that the connector cross-section is sufficient.

4. Select the desired Luxatemp MaxProtect color.
5. Fill the impression with Luxatemp MaxProtect. First dispense the material onto the occlusal surfaces and then fill up to the gingival areas. Keep excess material to a minimum.

Note: To avoid bubbles, keep the mixing tip immersed in the material throughout application.

6. No later than 40 seconds after the mixing process has started, place the filled impression with moderate pressure on the prepared teeth and fix it.
7. Monitor the setting process intra-orally (e.g. with a probe), as the curing process is influenced by the temperature in the mouth.
8. Remove the restoration from the mouth during the elastic phase, i.e. in the time period from 105 - 140 seconds after the mixing process has started.
9. Remove the oxygen-inhibited layer from the surface of the restoration (e.g. with alcohol).

Caution! When finishing the restoration, sanding dust can get into the eyes and respiratory tract.

- Wear a face mask and protective goggles.
 - Protect against grinding dust by proper ventilation – do not inhale the dust.
10. About 300 seconds after the mixing process is started, finish the restoration using a cross-cut grinder, flexible discs, etc.
 11. Check the occlusion and adjust accordingly before polishing.
 12. Polish with a rubber polisher or small goat's hair brush and a suitable polishing paste, if necessary.
 13. If necessary, coat the surface of the restoration with a light-curing high-gloss glaze.
 14. Thoroughly clean the preparation with water spray, dry it and thoroughly remove the separating medium. In doing so, ensure that the dentin retains a little residual moisture and does not become too dry.
 15. Slightly roughen the inner crown surfaces with a drill or sand blaster.
 16. Cement the restoration using a suitable permanent or temporary cement. Observe the manufacturer's instructions.

Note: Cements that contain eugenol may prevent the polymerization of resin-based luting cements during the definitive cementation.

Customizing and repairing a restoration

Note: Repairs should only be performed on small defects that do not negatively affect stability.

Caution! Grinding dust can get into the eyes and airways when working on or finishing the restoration.

- Wear a face mask and protective goggles.
 - Protect against grinding dust by proper ventilation – do not inhale the dust.
1. Customizing or repairing a fitted restoration: Roughen the restoration slightly using a grinder or a sandblaster at the appropriate points. Afterwards apply a bonding agent in accordance with the manufacturer's instructions.
 2. Apply Luxatemp MaxProtect to the prepared areas.
 3. After waiting for 300 seconds, finish the corresponding area in the usual manner.

Timing

Place into the mouth [s]*	0 - 40
Remove from the mouth [s]	105 - 140
Curing completed, finish the restoration [s]*	300

* The specified times are valid at a room temperature of 23 °C (73 °F) and a normal relative humidity of 50 %. Higher temperatures shorten, lower temperatures lengthen these times.

Interactions

- Residues of methacrylate-based materials may alter the setting behavior of the silicone impression and bite registration materials.
- If a restoration was prepared beforehand, carefully clean the preparation and the surrounding tissue with alcohol-soaked gauze and water spray before taking the impression.

Residual risks/side effects

There are no known side effects to date. The residual risk of hypersensitivity to components of the material cannot be ruled out.

Warnings/precautions

- For dental use only!
- Keep out of the reach of children!
- Avoid contact with the skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid contact with eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Product performance characteristics

Compressive strength \geq 300 MPa, flexural strength \geq 100 MPa

Storage/disposal

- Store in a dry place at room temperature (15 - 25 °C/59 - 77 °F)! Do not store in the refrigerator!
- Protect against exposure to direct sunlight!
- After the application, leave the used mixing tip on the Automix Cartridge as a seal!
- Do not use after the expiration date!
- Disposal must comply with national regulations.

Composition

Dental glass, EBPADMA, urethane acrylate oligomer, polyester, urethane methacrylate, SiO₂, TEDMA, HEMA, additives.

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