Flairesse Bleaching Gel CP 16%

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English

Product description

Flairesse Bleaching Gel CP is a gel containing 16 % carbamide peroxide (equivalent to 5.8 % hydrogen peroxide) for gentle tooth whitening. After being instructed by the dentist, the product can be used by the patient on their own under the guidance of the dentist.

Purpose

Gentle whitening of intrinsic and extrinsic discolorations.

Limitations of use

- Not for use on people under 18 years of age.
- Discoloration caused by metals (e.g. amalgam or silver pins) cannot be corrected by a whitening treatment.
- Do not use the material where there are known allergies to any of the components or contact allergies.
- Do not use the material during pregnancy or while breastfeeding.
- Do not use the material for heavy smokers unless they refrain from smoking during treatment.
- Do not use the material with heavy alcohol consumption.
- Do not use the material if periodontal diseases are present.
- Do not use the material on exposed dentine, defective filling margins or untreated caries.

Intended user

May only be supplied to dentists. The first application in each application cycle must always be reserved for a dentist or be performed under the direct supervision of a dentist if an equivalent level of safety can be guaranteed.

The product must then be provided to the consumer for the remaining application cycle.

Notes for use

Demineralized areas (white spots) may become more visible at the beginning of the treatment. The difference in color to the healthy enamel evens out over the course of the treatment.

Using the syringe

- The gel comes in a 5 ml syringe.
- The syringe must be closed with the corresponding cannula cap after each use.
- Only use it in combination with a bleaching tray.

Recommended use

Pre-treatment

1. Detailed examination of the specific situation in the patient's mouth to identify defective restorations, untreated caries or exposed dentine. These areas must be excluded from the bleaching treatment or properly taken care of before the treatment commences, otherwise the patient may experience pain.
2. Removal of external discoloration and tartar through professional tooth cleaning before treatment.

Note: If the gingiva becomes irritated, treatment must be delayed until the irritation has subsided so that any further sensitization can be avoided.

3. Determination of the type and extent of the corresponding discoloration and determination of whether bleaching treatment is indicated.
4. Determination of the initial tooth shade using a suitable shade guide (e.g. Vita classical®). Photographic documentation of the initial tooth shade can be useful to help the patient visualize the success of the treatment.

Note: Before commencing treatment, inform the patient that existing restorations do not whiten and that the shade of the surrounding enamel may look different afterwards.

5. The bleaching treatment must not be recommended if the patient is unable to follow the treatment safely.

Bleaching treatment

1. The bleaching tray should be manufactured using suitable materials in accordance with the manufacturer's instructions.
2. Open the syringe by twisting off the cap. Keep the cap in a safe place.

Note: The syringe must be resealed with the cap after use.

3. Apply a pinhead-sized amount to each reservoir of the splint to cover the areas to be treated.

Note: The correct filling quantity can be checked after inserting the tray by looking at the distribution of the material in the reservoir. The filling quantity may need to be adjusted during the next application.

4. Insert the filled bleaching tray. Excess material must be removed with a soft cloth or a soft toothbrush.
5. The tray must be worn for up to 8 hours according to the dentist's individual recommendation. Repeat the treatment daily until the desired tooth shade is achieved. Do not eat or drink during treatment.

Note: Interrupting treatment for several days is possible and does not jeopardize the subsequent continuation of treatment.

Notes: Avoid colored drinks and foods (e.g. wine, coffee, cola drinks) as well as acidic foods (e.g. citrus fruits) and drinks (e.g. sodas) during the entire treatment cycle.

6. Remove the tray and rinse the mouth several times with water. Do not swallow the water! If any gel remains on the teeth, it can be carefully removed with a soft toothbrush.
7. Clean the tray with cold water and a soft toothbrush. Store the tray in the tray box supplied.

Interactions

Carbamide peroxide can have a detrimental effect on the radical polymerization of plastics. Therefore, adhesive dental materials must not be used for at least two weeks after the bleaching treatment is completed.

Residual risks/side effects

There are no known side effects to date. The residual risk of hypersensitivity to components of the material cannot be ruled out.

Warnings/precautions

- Keep out of the reach of children!
- Avoid contact with skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid contact with eyes! Risk of blindness! In the event of accidental eye contact, rinse immediately and carefully with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- In case of exposure or suspicion: Get medical advice/attention.
- Contains alkali.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious adverse reactions involving this product must be reported to the manufacturer and to the responsible registration authorities.
- The patient must be instructed to avoid swallowing large quantities of gel.
- If there is risk of swallowing or inhaling the bleaching tray contents, treatment should not take place.
- Instruct the patient to book an appointment for a consultation if symptoms occur.

Product performance characteristics

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