DeltaFil Coat

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Instructions for use English

Product description

DeltaFil Coat is a light-curing protective varnish that protects GIC (glass ionomer cement) restorations from contamination and drying out during the first 24 h after treatment.

Intended purpose

Protection of a GIC restoration from moisture contamination and drying out during the first 24 h.

Limitations of use

Use only in combination with DeltaFil.

Indication

- Loss of dental hard tissue due to caries, fractures, tooth abrasion or developmental disorders
- Secondary prevention of caries

Contraindication

Do not use the material if there are known allergies to any of the components or contact allergies.

Patient target group

People who are receiving dental treatment.

Intended user

Dentist

Notes for use

DeltaFil Coat is light-sensitive. Open the bottle only right before use and close the bottle immediately afterwards.

Recommended use

1. Clean the GIC restoration after finishing. Remove excess water using cotton pellets or airflow. The surface should be dry, but not dried out.
2. Apply DeltaFil Coat immediately with a microbrush (see “Accessories”) in a homogeneous thin layer on all surfaces of the GIC restoration and adjacent areas.
3. Light-cure DeltaFil Coat with a suitable LED or halogen polymerization unit for at least 10 s at an exposure intensity of $\geq 1,000 \text{ mW/cm}^2$ or for 20 s at an exposure intensity of $\geq 600 \text{ mW/cm}^2$.
The oxygen inhibition layer can be removed with the help of a cotton pellet if necessary.

Interactions

Do not use DeltaFil Coat in combination with desensitizers and materials containing eugenol, as they may have a negative effect on the curing or adhesion of DeltaFil Coat.

Residual risks/side effects

- Please check the product for damage and visible contamination before use.
- Hypersensitivity and/or allergic reactions cannot be ruled out in individuals who are predisposed.
- Contact of DeltaFil Coat with the mucous membranes may cause skin irritation. Remove DeltaFil Coat from the affected area of the skin with a cotton pellet and rinse thoroughly with water.
- Improper use of the product can lead to excessive loss of dental hard tissue, eye irritation, respiratory tract irritation, skin or mucous membrane irritation and/or damage, choking, and infection.
- A low residual risk of unknown side effects cannot be ruled out.

Warnings/precautions

- For dental use only!
- Make sure the product is not swallowed! If it is accidentally swallowed, rinse the mouth thoroughly with water to eliminate any remaining material from the oral cavity. Do not induce vomiting!
- Avoid contact with the skin! In the event of accidental contact with the skin, immediately wash the affected area thoroughly with soap and water.
- Avoid contact with eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Observe the safety data sheet.
- Follow the manufacturer’s instructions for other products used with the material/materials.
- The manufacturer (productsafety@mtcompanies.com) and the responsible registration authorities must be notified of any serious incidents involving this product.

Product performance characteristics

Polymerization depth $\geq 2 \text{ mm}$ with a 10 s light-curing time when the light intensity of the device is $\geq 1000 \text{ mW/cm}^2$.

Polymerization depth $\geq 2 \text{ mm}$ with a 20 s light-curing time when the light intensity of the device is $\geq 600 \text{ mW/cm}^2$.

Storage/disposal

- Store at 15 – 25 °C (59 – 77 °F) in a dry and dark place!
- Do not use after the expiration date!
- Disposal must comply with national regulations.

Composition

UDMA, PPG-DMA, PEG-DMA, polymeric aminobenzoate derivative, MDP, additives.

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