Contax

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Activator: HEMA, Bis-GMA, MMHE, BPO, Additive.

Instructions for use

English

Product description

Contax is a light-curing, self-etching, and self-conditioning adhesive system for dentin and enamel. The Contax adhesive system can be used with and is compatible with light-, dual- and self-curing composites, compomers and ormocers. If the restoration is not light-cured in the later course of treatment, the activator must be used. Contax itself, however, must be light-cured, even if the Activator is used.

Intended purpose

Adhesive system for dental composites, compomers and ormocers to be used on dentin and enamel.

Limitations of use

Do not use the material if a sufficiently dry working area or the recommended application technique is not possible.

Indication

- Loss of tooth substance due to caries, trauma, tooth abrasion or development disorders
- Trauma and periodontitis requiring composite splints

Contraindication

- Do not use directly on the exposed pulp.
- Do not use the material if pulpitis is present.
- Do not use the material in cases of known allergies to any of the components, in particular benzoyl peroxide, or in the event of contact allergies.

Patient target group

People treated in the course of a dental procedure.

Intended user

Dentist

Notes for use

- Make sure that the primed and/or bonded surface does not become re-contaminated.
- Do not remove the oxygen inhibition layer caused by the application because this is important for the bonding process with the material applied on top.
- In deep cavities, material excesses can form at the margin between tooth and matrix («pooling effect»). Therefore, if possible, use a matrix only after applying the bond.
- Do not interchange lids of the bottles because this can lead to a cross-contamination of the liquids. For better differentiation, the lids are color-coded: Red = Primer; Black = Bond; White = Activator.
- In order to prevent premature polymerization due to ambient light, the material should be placed on the mixing pad immediately prior to the application.
- The Contax Activator must be used if self- or dual-curing restoration materials are used. Use of the Contax Activator is not necessary if light-curing restoration materials are used. Contax must be light-cured in any case.
- Light-curing units should have an output of 450 nm and should be checked regularly. The light intensity should be at least 600 mW/cm². Place the light-curing unit as close as possible to the material.
- DMG recommends the use of a rubber dam.
- If the material is applied into the patient's mouth with a single use applicator, the single use applicator must be used for this one patient only for hygienic reasons.

Recommended use

1. Prepare the cavity in accordance with the general rules for the adhesive technique. Bevel the enamel edges. Protect pulp in deep cavities with suitable materials.

Note: If additional enamel etching is desired, this can be performed according to the manufacturer's instructions. The bond strength of Contax is not affected.

2. Put one drop of Contax Primer onto the mixing pad or directly onto the single-use brush and work into the tooth structure for 20 seconds.

Note: After the application of the Contax Primer, the surface to be treated must appear evenly moist. If this is not the case, apply Contax Primer again. It is not necessary to blow air on the Contax Primer.

3. Prepare the bond as follows:
 - For self- or dual-curing restoration materials: Mix one drop of Contax Bond and one drop of Contax Activator in the mold of the mixing pad for approximately 5 seconds.
 - For light-curing restoration materials: Put one drop of Contax Bond on the mixing pad or directly onto a new single-use brush. Contax Activator is not necessary.
4. Apply the bond or bond-activator mixture to the whole primed preparation.
5. Work the bond or bond-activator mixture for 20 seconds.
6. Remove excess material.
7. Thin out the bond or bond-activator mixture briefly with oil-free air until the surface appears evenly moist.
8. Light-cure Contax for 20 seconds.
9. Apply the restoration material according to the manufacturer's instructions.

Timing

Apply the Contax Primer and work into the tooth	20 s
For light-curing materials: Apply the Contax Bond and work into the tooth Remove excess and blow with compressed air Light-curing	20 s 20 s
For self- and dual-curing materials: Mix Contax Bond and Contax Activator 1: 1 Apply and work into the tooth Remove excess and blow with compressed air Light-curing	5 s 20 s 20 s

Interactions

Do not use together with substances which inhibit polymerization. This is especially valid for products which contain eugenol, such as temporary cements or filling materials.

Residual risks/side effects

There are no known side effects to date. The residual risk of hypersensitivity to components of the material cannot be ruled out.

Warnings/precautions

- Keep out of the reach of children!
- Avoid contact with the skin! In the event of accidental skin contact immediately wash the affected area thoroughly with soap and water.
- Avoid contact with eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Product performance characteristics

SBS bonding values on unetched tooth substance measured in accordance with ISO 29022:2013 are ≥ 15 MPa

Storage/disposal

- Store in a dry place at 2 to 25°C/36 to 77°F!
- Do not use after the expiration date.
- Close packaging immediately after use.
- Dispose of in accordance with national regulations.

Composition

Primer: Water, maleic acid, sodium fluoride.

Bond: HEMA, bis-GMA, GDMA, glycidyl methacrylate-modified polyacrylic acid, additives.

Activator: HEMA, bis-GMA, MMHE, BPO, additives.

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Activator: HEMA, 비스-GMA, MMHE, BPO, 첨가제.

Packaging

Starter Pack	
1 Bottle @ 5 ml Primer, 1 Bottle @ 5 ml Bond, 50 Single-use Brushes White, 50 Single-use Brushes Red, 1 Mixing palette	REF 220263
Starter Pack + activator	
1 Bottle @ 5 ml Primer, 1 Bottle @ 5 ml Bond, 1 Bottle @ 5 ml Activator, 50 Single-use Brushes White, 50 Single-use Brushes Red, 1 Mixing palette	REF 220282
Refill	
1 Bottle @ 5 ml Primer, 1 Bottle @ 5 ml Bond, 1 Bottle @ 5 ml Activator	REF 220281 REF 220284 REF 220283
Accessories	
50 Single-use Brushes White	REF 109320