# Ecosite Bulk Fill

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English

### Product description

Ecosite Bulk Fill is a light-curing, radiopaque restorative material developed especially for use in the posterior region.

### Intended purpose

- Dental fillings in the posterior region (Classes I and II, including replacement of individual cusps)
- Restorations of Class V (cervical caries, root erosions, wedge-shaped defects)
- Core build-up
- Extended fissure sealing on molars and premolars
- Restorations in deciduous teeth

### Limitations of use

Application of the material is contraindicated if dry isolation or the recommended application technique is not possible.

### Indication

- Loss of tooth substance due to caries, trauma, tooth attrition/abrasion or development disorders
- Use in molars and premolars with small carious defects located in pronounced deep fissures: Extended fissure sealing

### Contraindication

- Do not use directly on the exposed pulp.
- Do not use the material where there are known allergies to any of the components or contact allergies.

### Patient target group

Persons receiving treatment as part of a dental procedure.

### Intended user

Dentist

### Notes for use

- For optimum polymerization, position the light-curing unit as close to the restorative material as possible and observe the recommendations relating to the light-curing time and the light intensity (see table). Observe the manufacturer's instructions for the light-curing unit used.
- Follow the manufacturer's instructions for other materials that are used with Ecosite Bulk Fill.
- If the material is applied to the patient's mouth with a single use applicator, the single use applicator must be used for this one patient only for hygienic reasons.

### Recommended use

1. Clean the tooth. Remove all cleaning residue with water spray. The use of a rubber dam is recommended.
2. Prepare the cavity in accordance with the general rules for adhesive procedures. Bevel the enamel edges. Ensure that the areas close to the pulp are protected.
3. Condition the dental enamel and apply the bonding agent according to the manufacturer's instructions for the material used. DMG advises using the universal adhesive Ecosite Bond.
4. If necessary, apply a base of low-viscosity composite or compomer. We recommend using a matrix.
5. Insert Ecosite Bulk Fill with a maximum layer thickness of 5 mm into the cavity. Apply taller restorations one layer at a time. Light-cure each layer separately.

Light intensity	$\geq 1000$ mW/cm <sup>2</sup>	$\geq 600$ mW/cm <sup>2</sup>
Light-curing time	20 s	40 s

**Note:** Additional buccal or lingual light-curing is advisable. Remove metal matrices beforehand.

6. Removal of any excess material, contouring, and finishing are done using diamond finishing tools and flexible polishing disks of decreasing grain size. Ecosite Bulk Fill can be given a high shine using polishing paste together with polishing cups or disks.

### Handling of the Safetip Dispenser Pro

1. Pull the quick-lock mechanism all the way back.
2. Insert the Safetip into the Safetip Dispenser Pro, and then guide the quick-lock mechanism back into place until the Safetip is clearly engaged.
3. Turn the Safetip dispensing nozzle in the desired direction, remove the cap from the Safetip and dispense the material by pressing evenly. Do not exert too much force when pressing.
4. After dispensing, pull the quick-lock mechanism back again and remove the Safetip.
5. The Safetip Dispenser Pro should be cleaned and disinfected after use.

### Disinfection of the Safetip Dispenser Pro

The Safetip Dispenser Pro is reusable after disinfection, although it should be replaced if visible signs of wear appear.

The Safetip Dispenser Pro is suitable for steam sterilization in autoclaves. The procedure must be carried out at 134 °C/3 bar for 5 minutes.

### Handling of the Safetip Dispenser

1. Pull spring piston backwards.
2. Place the Safetip on the front of the Safetip Dispenser.
3. Turn the dispensing nozzle in the desired direction, remove the cap from the Safetip and dispense the material by pressing evenly. Do not exert too much force when pressing.
4. After dispensing, pull the spring piston backwards, remove Safetip.
5. The Safetip Dispenser should be cleaned and disinfected after use.

### Disinfection of the Safetip Dispenser

The Safetip Dispenser is reusable after disinfection, although it should be replaced if visible signs of wear appear.

Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch Institute). The Safetip Dispenser is also autoclavable.

### Interactions

- Avoid materials containing eugenol, moisture and oily air, as these may inhibit polymerization at the contact area.
- The use of mouthwashes, plaque indicators and chlorhexidine may lead to discoloration.

### Residual risks/side effects

There are no known side effects to date. The residual risk of hypersensitivity to components of the material cannot be ruled out.

### Warnings/precautions

- Keep out of the reach of children!
- Avoid contact with the skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

### Product performance characteristics

The product complies with DIN EN ISO 4049:2019, Type 1, Class 2, Group 1

Polymerization depth  $\geq 4$  mm with a 20 second light-curing time when the light intensity of the lamp is  $\geq 1000$  mW/cm<sup>2</sup>

Polymerization depth  $\geq 4$  mm with a 40 second light-curing time when the light intensity of the lamp is  $\geq 600$  mW/cm<sup>2</sup>

### Storage/disposal

- Store in a dry place at 2 to 25 °C (36 to 77 °F)
- Disposal must comply with national regulations.

## Composition

Dental glass, EBPADMA, SiO<sub>2</sub>, bis-GMA, NPG2PODA, UDMA, additives. Inorganic filling materials: approx. 65 vol% (0.02 - 0.7 µm).

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