Flairesse Foam

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Instructions for use English

Product description

Prophylaxis foam for final treatment as part of professional tooth cleaning (PTC) and for integration into an individual prevention plan. Flairesse Foam is available in three different flavours.

Intended purpose

Supports remineralization of enamel after professional tooth cleaning.

Limitations of use

Do not use the material if the patient's control over their swallowing reflex is not guaranteed.

Indications

Teeth with a high caries risk

Contraindications

Do not use the material where there are known allergies to any of the components or contact allergies.

Patient target group

People treated in the course of a dental procedure.

Intended users

Dentist/dental personnel

Notes for use

- Shake well for at least 30 seconds before each use.
- If the material is applied into the patient's mouth with a single use applicator, the single use applicator must be used for this one patient only for hygienic reasons.
- The dispenser is fitted with an tamper-proof seal. Make sure that the tamper-proof seal is intact before first use.
- When using the dispenser for the first time, snap off the nozzle cap by gently pushing it forwards.

Recommended use

Caution! Risk of swallowing if saliva suction is insufficient.

- Saliva must be suctioned away thoroughly during treatment.

Note: Activating the nozzle once will produce enough material for one dental arch.

1. Hold the dispenser vertically with the nozzle pointing down into the application tray.
2. Activate the nozzle to express the foam into the application tray.
3. Dry the tooth surfaces.
4. Apply the application tray to the dental arch.
5. Leave the application tray on the dental arch for 1 minute.
6. Remove the application tray at the end of the response time.
7. Get the patient to spit out the excess prophylaxis foam.

Note: For guaranteed effectiveness, inform the patient that they should not do the following for 30 minutes after using the prophylaxis foam:

- eat
- drink
- rinse their mouth

Interactions

No interactions are currently known.

Residual risks/side effects

- We place great importance on the quality of our products. However, please check the product for damage and visible contamination before use.
- Swallowing excessive amounts of the material may cause nausea.
- Hypersensitivity and/or allergic reactions cannot be ruled out in individuals who are correspondingly predisposed.

Warnings/precautions

- For dental use only.
- Avoid contact with eyes! In the event of accidental contact with the eyes, immediately rinse the eyes thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Do not spray into naked flames.
- Pressurised container. Do not open with force. Do not pierce or burn.
- Do not swallow.
- Serious incidents involving this product must be reported to the manufacturer (productsafety@mtcompanies.com) and to the responsible registration authorities.

Product performance characteristics

Fluoride content: approx. 12,300 ppm

Storage/disposal

- Store in a dry place at 15 to 25 °C(59 to 77 °F)!
- Do not store in the refrigerator.
- Keep away from direct sunlight.
- Do not use after the expiration date.
- Dispose of in accordance with national regulations.

Composition

Water (approx. 88 %), sodium fluoride (< 3 %), additives(< 10 %)

Free from aspartame, gluten and saccharin.

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