Honigum Putty

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Honigum Putty: SiO₂, Polyvinylsiloxan, Vinyl-Polydimethylsiloxan, Vaseline, Aluminiumsilikat, Additive, Pigmente, Katalysatoren.

Honigum Putty Soft: SiO₂, Polyvinylsiloxan, Paraffinöl, Vaseline, Polydimethylsiloxan, Additive, Pigmente, Katalysatoren.

Honigum Putty Soft Fast: SiO₂, Polyvinylsiloxan, Paraffinöl, Vaseline, Polydimethylsiloxan, Additive, Pigmente, Katalysatoren.

Honigum Putty Rigid Fast: SiO₂, Polyvinylsiloxan, Paraffinöl, Vinyl-polydimethylsiloxan, Vaseline, Additive, Pigmente, Katalysatoren.

Instructions for use

English

Product description

Honigum Putty is an initial impression material based on A-silicones. The material is available as a hand-mixed variant and in the ready-to-use pre-filled MixStar Cartridge, which is suitable for use in the MixStar eMotion and other commercially available mixing machines.

Intended purpose

Impressions for the fabrication of indirect restorations, such as crowns, bridges, inlays and onlays.

Limitations of use

Do not use the material for monophasic impressions.

Indication

All dental situations requiring final impressions, e.g. tooth loss, caries and trauma.

Contraindication

Do not use the material where there are known allergies to any of the components or contact allergies.

Patient target group

Persons receiving treatment as part of a dental procedure.

Intended user

Dentist, dental assistant staff

Notes for use

- The recommended times can be found in the section headed "Timing".

- In order to achieve an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are available as refills.
- Honigum Putty can be used in all conventional impression trays.
- In order to improve the adhesion of the material to metal or plastic trays, a tray adhesive can be applied (e.g. DMG Tray Adhesive).

Using the MixStar Cartridge in the MixStar eMotion

(Fig. see “Handling”)

Follow the instructions for use for the “MixStar eMotion” unit!

1. Slide the locking mechanism upwards in order to open the MixStar Cartridge. Do not remove the locking mechanism from the MixStar Cartridge [Fig. 1].
2. Remove the seal from the MixStar Cartridge and discard. Attach a new mixing tip [Fig. 2].
3. Secure the mixing tip by sliding the locking mechanism into the locked position [Fig. 3].
4. Insert the MixStar Cartridge into the MixStar eMotion [Fig. 4]. A brief acoustic signal confirms correct insertion of the cartridge.

Note: Prior to using a new cartridge for the first time, discard approx. the first 3 cm of the ejected material.

Note: After the application, leave the used mixing tip on the MixStar Cartridge as a seal.

Using the MixStar Cartridge in mixing units from other manufacturers

Follow the instructions for use for the mixing unit used!

- First insert the MixStar Cartridge into the mixing unit without the mixing tip. Otherwise appropriate mixing of the material cannot be guaranteed!
- In this case proceed as described in the section “Changing the mixing tip.”

Changing the mixing tip

Follow the instructions for use for the mixing unit used!

Note: Attach a new mixing tip each time before mixing.

1. Slide the locking mechanism upwards.
2. Remove the old mixing tip as well as any hardened material blocking the cartridge openings.
3. Place the mixing tip securely on the corresponding MixStar Cartridge opening.
4. Lock the mixing tip in place with the locking mechanism.

Handling the hand-mixing system

Note: Please avoid the use of latex gloves. DMG recommends the use of PVC or PE-based gloves.

1. Always take equal amounts of the base paste and the catalyst paste from the jars using the colored spoon and the white spoon respectively.
2. Mixing: Using your fingertips, knead the base paste and the catalyst paste at a ratio of 1:1 for 30 seconds at room temperature until the color is homogeneous.

Note: Use gloves or foil to avoid contamination.

3. Close the jars again immediately after use! Do not mix up the container lids!

Recommended use

Double mixing impression

1. Fill the impression tray with Honigum Putty. When doing so, hold the impression tray at a slight angle beneath the mixing tip.

Note: To avoid air bubbles, the end of the mixing tip should remain immersed in the material throughout application.

2. Inject Honigum Light around the preparation. Follow the corresponding instructions for use!

Note: Filling of the impression tray and injection should be concluded at the same time, as the higher temperature in the mouth causes the injected material to set more quickly than the material in the impression tray.

3. Position the filled impression tray slowly and gently in the mouth within the working time and take the impression of the preparation.
4. Remove the impression from the mouth once the setting time has ended (see “Removal”).

Corrective impression

Note: Do not allow more than 30 min to pass between completion of the initial impression and the final impression and do not clean the impression with alcohol, as there is then no guarantee of a secure bond between the two impression materials.

1. Cut generous drainage channels into the initial impression for the wash and remove imperfections.
2. After testing by repositioning, rinse the impression thoroughly with water and dry.
3. Apply Honigum Light around the preparation and onto the initial impression. Follow the corresponding instructions for use!
4. Position the filled impression tray slowly and gently in the mouth within the working time, initially exerting brief pressure.
5. Remove the impression from the mouth once the setting time has ended (see “Removal”).

Removal

1. In the area where the teeth have been prepared, remove the impression parallel to the longitudinal axis of the teeth.
2. After removal, proceed as described in the section “Cleaning and disinfection of the final impression.”

Cleaning and disinfection of the final impression

- Only disinfect the final impression using dipping solutions.
- DMG recommends the use of a 2 % glutaraldehyde solution.
- Ensure that the disinfectant used is indicated for use with addition-cured silicone impression material.
- Follow the manufacturer’s instructions for the disinfectant solution precisely!
- Leave the final impression to dry and store until casting (see “Storage of the impression”).
- For more information, see the statement issued by the German Society for Dental and Oral Medicine (DGZMK). Comments of the DGZMK 3/93 V 1.0; www.dgzmk.de
- ADA recommends following the advice of the Council on Dental Materials, Instruments and Equipment. JADA 1991;122(3):110; www.ada.org

Casting and galvanization

- After removing the impression from the mouth, wait at least 30 min before casting.
- DMG recommends using a special purpose hard plaster (ISO 6873:2013; Type 3, e.g. Moldano, Bayer) for the cast and an ultra hard dental plaster for the models (ISO 6873:2013; Type 4, e.g. Fujirock II, GC).
- The impression can be copper or silver-plated.

Storing the impression

- Store the impression at 15 – 25 °C (59 – 77 °F) in a dry place!
- Protect from exposure to direct sunlight!
- The impression is stable and can be cast even after months without loss of accuracy.

Timing

Honigum Putty				
		Soft	Soft Fast	Rigid Fast
Mixing time [s]		30	30	30
Working time including mixing time* [s]	≤ 105	≤ 165	≤ 105	≤ 90
Setting time in the mouth [s]	≥ 195	≥ 255	≥ 120	≥ 105

* The specified times are valid at a room temperature of 23 °C and a normal relative humidity of 50 %. Higher temperatures shorten, lower temperatures lengthen these times. The working time describes the period from when mixing begins to the time by which the impression tray must be positioned in the patient’s mouth.

Interactions

- Rinse off any residual retraction solution thoroughly with water, otherwise this may lengthen the setting time in some instances.
- Never allow the impression to come into contact with organic solvents or liquids containing solvents as this may result in a swelling of the material and may therefore lead to inaccuracies in the cast.
- Do not allow the impression material to come into contact with latex gloves. DMG recommends the use of PVC or PE-based gloves.
- Do not mix Honigum Putty with condensation curing silicone or polyether impression materials.
- Leftover methacrylate-based materials may alter the setting behavior of the silicone impression materials. If a restoration was prepared beforehand, carefully clean the preparation and the surrounding tissue with alcohol-soaked gauze and water spray before taking the impression.

Residual risks/side effects

- We place great importance on the quality of our products. However, please check the product for damage and visible contamination before use.
- There are no known side effects to date.
- No cases of hypersensitivity or allergic reactions are known to date. However, reactions to components of the material cannot be ruled out.

Warnings/precautions

- For dental use only!
- Avoid contact with the eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.

- Follow the manufacturer’s instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer (productsafety@mtcompanies.com) and to the responsible registration authorities.

Product performance characteristics

Meets the requirements of ISO 4823:2021, Type 0: putty consistency

Storage/disposal

- Store in a dry place at 15 to 25 °C (59 to 77 °F).
- Protect from exposure to direct sunlight!
- After the application, leave the used mixing tip on the MixStar Cartridge as a seal!
- Do not use after the expiration date!
- Disposal must comply with national regulations.

Composition

Honigum Putty: SiO₂, polyvinylsiloxane, vinyl polydimethylsiloxane, Vaseline, aluminum silicate, additives, pigments, catalysts.

Honigum Putty Soft: SiO₂, polyvinylsiloxane, paraffin oil, Vaseline, polydimethylsiloxane, additives, pigments, catalysts.

Honigum Putty Soft Fast: SiO₂, polyvinylsiloxane, paraffin oil, Vaseline, polydimethylsiloxane, additives, pigments, catalysts.

Honigum Putty Rigid Fast: SiO₂, polyvinylsiloxane, paraffin oil, vinyl polydimethylsiloxane, Vaseline, additives, pigments, catalysts.

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