LuxaPick-up

Inhalt | Contents

Gebrauchsinformation – DEUTSCH	4
Instructions for use – ENGLISH	12
Mode d'emploi – FRANÇAIS	20
Istruzioni d'uso – ITALIANO	28
Instrucciones de uso – ESPAÑOL	36
Instruções de uso – PORTUGUÊS	44
Gebruiksaanwijzing – NEDERLANDS	52
Brugsanvisning – DANSK	60
Användarinstruktioner – SVENSKA	68
Instrukcja użycia – POLSKI	76
Информация по применению – РУССКИЙ	85
使用说明-简体中文	94
使用説明書-日本語	101
사용 설명서-한국의	109

Produktbeschreibung

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Product description

LuxaPick-up is a self-curing relining composite based on multifunctional methacrylates. The material is available in the Automix cartridge.

Indications

- Fixation of secondary parts and connection elements in dentures/prostheses
- Relining of mucosa-supported dentures
- Modification of interim dentures
- Repair of prostheses
- Fabrication of gingival masks
- Customization of prostheses to fabricate natural red-white esthetics, especially in the case of implant-borne dentures

Contraindications

Do not use the material in cases of allergies to any of the ingredients or, as can occur in rare cases, in the event of contact allergies.

Basic safety information

- For dental use only.
- Keep out of the reach of children.
- Avoid contact with the skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.

Side effects

There are no known systemic side effects to date. Hypersensitivity to components of the material or contact allergies cannot be excluded in individual cases.

Timing

0:00–0:45 min	Place into the mouth
2:00–3:00 min	Remove from the mouth
6:00–7:00 min*	Setting completed

*The indicated times apply at a room temperature of 23°C/73°F and a normal relative humidity of 50%. Higher temperatures shorten, lower temperatures increase these times.

Recommended use

Fixation of secondary parts in prostheses – chairside

1. Shape a circular undercutting groove in the base of the prosthesis, to create space for the secondary parts.
 - 1.1 A 5 mm indentation and a groove can be formed.
2. Clean and dry the prosthesis.
3. Fix the secondary part on the implant.

4. Apply Luxatemp-Glaze & Bond according to the manufacturer's instructions to the areas of the prosthesis to be joined with LuxaPick-up and allow to act.
5. Light-cure Luxatemp-Glaze & Bond according to the manufacturer's information.

Note: Ensure that the conditioned surface is not contaminated.

6. Apply LuxaPick-up in the restoration and insert and fix into place using moderate pressure no later than 45 s after mixing has begun.

Note: The secondary part should be encapsulated with sufficient material. Remove excess material with a suitable instrument.

7. Allow LuxaPick-up to harden completely.

8. Remove the restoration from the mouth.
 9. Trim junctions and grind down excess.
 10. Polish abraded areas, or seal with a suitable conditioner, and then cure according to the manufacturer's instructions.
3. Check whether the secondary part/connection element fits into the corresponding groove without touching the base of the prosthesis.
 4. Clean and dry the prosthesis.
 5. Apply Luxatemp-Glaze & Bond according to the manufacturer's instructions to the areas to be joined with LuxaPick-up and allow to act.

Fixation of secondary parts in prostheses – labside

1. Shape a circular undercutting groove in the base of the prosthesis, to create space for the secondary parts.
 - 1.1 A 5 mm indentation and a groove can be formed.
 2. Fix the secondary parts on the implant.
6. Light-cure Luxatemp-Glaze & Bond according to the manufacturer's information.
- Note:** Ensure that the conditioned surface is not contaminated.
7. Apply LuxaPick-up in the restoration and insert and fix into place using moderate pressure no later than 45 s after mixing has begun.

Note: The secondary part should be encapsulated with sufficient material. Remove excess material with a suitable instrument.

8. Allow LuxaPick-up to harden completely.
9. Remove restoration from the model.
10. Trim junctions and grind down excess.
11. Polish abraded areas, or seal with a suitable conditioner, then cure according to the manufacturer's instructions.

Relining or repair of mucosa-supported dentures – chairside

1. Abrade areas which are to be joined using LuxaPick-up.
2. If there are any cracks or broken-off regions, abrade the affected areas.
3. Clean and dry the prosthesis.

4. Apply Luxatemp-Glaze & Bond according to the manufacturer's instructions to the areas of the prosthesis to be joined with LuxaPick-up and allow to act.
5. Light-cure Luxatemp-Glaze & Bond according to the manufacturer's information.

Note: Ensure that the conditioned surface is not contaminated.

6. Apply LuxaPick-up in the restoration and insert and fix into place using moderate pressure no later than 45 s after mixing has begun.

Note: Remove excess material with a suitable instrument.

7. Remove the restoration from the mouth during the elastic phase, i.e. in the time period from 2:00 to 3:00 min after the start of mixing.

8. Allow LuxaPick-up to harden completely.
9. Trim junctions and grind down excess.
10. Polish abraded areas or seal with a suitable conditioner and then cure according to the manufacturer's instructions.

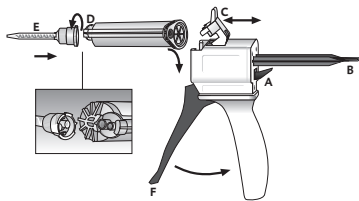
Relining or repair of mucosa-supported dentures – labside

1. Plaster the prosthesis including the function impression in a fixator. As soon as the plaster is partially hardened, create a counter for the subsequent bonding.
2. Let the plaster harden completely as per the manufacturer's instructions.
3. Open the fixator and remove the prosthesis.
4. Remove the remnants of the impression material from model and prosthesis.

5. Abrade areas of the prosthesis to be joined using LuxaPick-up. Apply Luxa-temp-Glaze & Bond to the abraded areas according to the manufacturer's instructions and allow to act.
6. Clean and dry the prosthesis.
7. Paint plaster-resin insulation carefully onto the model and counter.
8. Affix the prosthesis to plaster counter (e.g. with sticky wax).
9. Apply LuxaPick-up to the areas to be relined and mount it in the fixator at the latest 45 s after mixing has started.
10. Remove any large excess with a spatula. Then polymerize in a pressure pot at 45°C for 10 min.
11. Deflask the prosthesis after removal, trim junctions and grind down any excess using a plastic milling cutter.

- Polish abraded areas or seal with a suitable conditioner and then cure according to the manufacturer's instructions.

Using the Automix cartridge

Inserting the cartridge

- Push up the lever [A] at the back of the Automix-Dispenser and pull back the slide [B] all the way.
- Lift up the plastic lever [C], insert the cartridge and lock it using the plastic lever [C].

Note: Make sure that the notches on the cartridge line up with those on the Automix-Dispenser.

Attaching a mixing tip

- Remove the cartridge cap [D] or used mixing tip after turning it 90° counterclockwise, and discard it.
- Attach a new mixing tip [E].

Note: In order to achieve an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are also available in refill packs.

Note: Make sure that the notches on the mixing tip line up with those on the cartridge.

- Twist the mixing tip clockwise through 90° to lock it.

Applying material

- When using a cartridge for the first time, squeeze out an approximately pea-sized amount of material and discard it.

- ▶ Mix the material in the mixing tip using the lever [F] on the Automix-Dispenser and apply directly.

Note: After application, leave the used mixing tip on the cartridge as a seal!

Removing the cartridge

1. Push up the lever [A] at the back of the Automix-Dispenser and pull back the slide [B] all the way.
2. Lift up the plastic lever [C] and remove the cartridge.

Disinfection of the Automix-Dispenser

The Automix-Dispenser can be reused after disinfection, although it should be replaced if there are visible signs of wear.

Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch Institute). The Automix-Dispenser is also autoclavable.

Storage and shelf life

- ▶ Store at room temperature (15–25°C/59–77°F) in a dry place.
- ▶ Do not store in the refrigerator.
- ▶ Keep away from direct sunlight.
- ▶ Do not use after the expiration date.

Composition

Glass filler material in a matrix of multifunctional methacrylates; catalysts, stabilizers, and additives. Does not contain methyl methacrylate or peroxides. Filler content: 43 wt.% = 24 vol.%. (0.02 to 1.5 µm)

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DMG Chemisch-Pharmazeutische Fabrik GmbH