# O-Bite Scan

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## Instructions for use English

### Product description

A-silicone-based, scannable bite registration material with high final hardness. The material is suitable for use with all scanners designed for scanning impression materials (e.g. the 3Shape D700 dental scanner). No additional surface treatment is required.

### Intended purpose

- Bite registration
- Material for application to bite forks on face-bows
- Key material, e.g. for intraoral incisal pin records

### Limitations of use

Do not use O-Bite Scan for taking conventional impressions.

### Indication

Dental cases that require impressions of both jaws, e.g. tooth loss

### Contraindication

Do not use the material if there are known allergies to any of the components or contact allergies.

### Patient target group

People who are receiving dental treatment.

### Intended user

Dentist, dental assistant staff

### Notes for use

- If the material is applied directly in the patient's mouth using a single-use applicator, the single-use applicator must be used only on this one patient for hygienic reasons.
- To achieve an optimum mixing result, use the mixing tips available from DMG. All mixing tips are also available in refill packs.

## Handling the Automix Cartridge (Fig. see "Handling")

### Inserting the Automix Cartridge

1. Push up the lever [A] at the back of the Automix Dispenser and pull back the slide [B] all the way.
2. Lift up the plastic latch [C], insert the Automix Cartridge and lock it using the plastic latch [C].

**Note:** Make sure that the notches on the Automix Cartridge line up with those on the Automix Dispenser.

### Attaching a mixing tip

1. Remove the cartridge cap [D] or used mixing tip after turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

**Note:** Make sure that the notches on the mixing tip line up with those on the Automix Cartridge.

3. Turn the mixing tip 90° clockwise to lock it.

### Applying the material

When using an Automix Cartridge for the first time, squeeze out a pea-sized amount of material and discard it.

1. Mix the material in the mixing tip using the lever [F] on the Automix Dispenser and apply directly.

**Note:** After use, clean the used mixing tip with a standard disinfectant solution and leave it on the Automix Cartridge to act as a cap!

### Removing the Automix Cartridge

1. Push up the lever [A] at the back of the Automix Dispenser and pull back the slide [B] all the way.
2. Lift up the plastic latch [C] and remove the Automix Cartridge.

### Disinfecting the Automix Dispenser

- The Automix Dispenser can be reused after disinfection, although it should be replaced if there are visible signs of wear.
- Disinfection can be carried out using a commercially available immersion disinfectant. DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch Institute).

## Recommended use

### Taking a bite registration

1. Apply the bite registration material directly to the occlusal surfaces. If necessary, cover the prepared areas first.

**Note:** It is usually sufficient to apply the material to the teeth of the upper or lower dental arch. In the anterior region, it is advisable to apply the material to both dental arches. A layer of material approximately 5 mm thick should be sufficient.

2. Allow the patient to adopt the desired occlusion.
3. After 90 s, carefully remove the finished bite registration from the mouth.

## Processing the bite registration

**Note:** The material has a high final hardness and can be machined using a rotary instrument (e.g. a grinder). Immediately after removal, any excess material can also be removed with a scalpel.

1. Remove any excess material that extends beyond the molded occlusal surfaces.
2. Remove any material fringes that extend into the interproximal spaces.

## Disinfecting the bite registration model

Disinfect bite registrations using immersion solutions only.

- DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch Institute) or the EPA (Environmental Protection Agency). Observe the manufacturer's instructions!
- Ensure that the disinfectant solution used is indicated for use with addition-curing impression silicones.

## Timing

<b>O-Bite Scan</b>	
Working time including mixing time* [s]	≤ 30
Setting time in the mouth [s]	≥ 90

\* The specified times are valid at a room temperature of 23 °C (73 °F) and a normal relative humidity of 50 %. Higher temperatures shorten these times; lower temperatures lengthen these times.

## Interactions

- Do not allow the material to come into contact with organic solvents or liquids that contain solvents as this can cause the material to swell and lead to inaccuracies.
- Avoid contact with retraction solutions as retraction solution residues can increase the setting time in some cases. Remove retraction solutions by thorough spraying and rinsing.
- Do not combine the material with condensation-curing silicone or polyether impression materials.
- Do not allow the material to come into contact with latex gloves as this inhibits the setting process. In the event of prior contact with latex gloves, wash hands carefully. The use of PVC or PE-based gloves is recommended.

## Residual risks/side effects

- Please check the product for damage and visible contamination before use.
- Hypersensitivity and/or allergic reactions cannot be ruled out in individuals who are predisposed.
- A low residual risk of unknown side effects cannot be ruled out.

## Warnings/precautions

- For dental use only.
- Avoid contact with the skin! In the event of accidental contact with the skin, immediately wash the affected area thoroughly with soap and water.
- Avoid contact with eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products used with the material/materials.
- Observe the safety data sheet.
- The manufacturer (productsafety@mtcompanies.com) and the responsible registration authorities must be notified of any serious incidents involving this product.

## Product performance characteristics

Product complies with ISO 4823:2025, Type B

## Storage/disposal

- Store at 15 – 25 °C (59 – 77 °F) in a dry place.
- Do not store in the refrigerator or freezer.
- Keep away from direct sunlight.
- After use, clean the used mixing tip with a standard disinfectant solution and leave it on the Automix Cartridge to act as a cap!
- Do not use after the expiration date!
- Disposal must comply with national regulations.

## Composition

SiO<sub>2</sub>, vinyl silicones, additives, pigments, catalysts, hydrogen silicone.

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