Silagum Putty Silagum Light

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English

Product description

Silagum Putty initial impression material and Silagum Light wash impression material, both based on A-silicones, for all two-step impression techniques. Silagum Light is available in Automix Cartridges.

Intended purpose

Impressions for the fabrication of indirect restorations, such as crowns, bridges, inlays and onlays.

Limitations of use

Do not use the material for monophasic impressions.

Indication

All dental situations requiring final impressions, e.g. tooth loss, caries and trauma.

Contraindication

Do not use the material if there are known allergies to any of the components or contact allergies.

Patient target group

People who are receiving dental treatment.

Intended user

Dentist, dental assistant staff

Notes for use

- The recommended times can be found in the chapter headed “Timing.”
- If the material is applied directly in the patient’s mouth using a single-use applicator, the single-use applicator must be used only on this one patient for hygienic reasons.
- To achieve an optimum mixing result, use the mixing tips available from DMG. All mixing tips are available as refill packs. The Intraoral Tips are to be used for direct, intrasulcular application around the preparation.
- Silagum Putty can be used in all conventional impression trays. In order to improve the adhesion of the material to metal or plastic trays, a tray adhesive can be applied (e.g. DMG Tray Adhesive).

Handling the Automix Cartridge (Fig., see “Handling”)

Inserting the Automix Cartridge

1. Push up the lever [A] at the back of the dispensing unit and pull back the slide [B] completely.
2. Lift up the plastic latch [C], insert the Automix Cartridge and lock it using the plastic latch [C].

Note: Make sure that the notches on the Automix Cartridge line up with those on the dispensing unit.

Attaching a mixing tip

1. Remove the cartridge cap [D] or used mixing tip after turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

Note: Make sure that the notches on the mixing tip line up with those on the Automix Cartridge.

3. Turn the mixing tip 90° clockwise to lock it.
4. Attach an Intraoral Tip [F], if needed.

Applying the material

- When using an Automix Cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- Mix the material in the mixing tip using the lever [G] on the dispensing unit and apply directly.

Note: After use, clean the used mixing tip with a standard disinfectant solution and leave it on the Automix Cartridge to act as a cap!

Removing the Automix Cartridge

1. Push up the lever [A] at the back of the dispensing unit and pull back the slide [B] completely.
2. Lift up the plastic latch [C] and remove the cartridge.

Disinfecting the Automix Dispenser

The Automix Dispenser can be reused after disinfection, although it should be replaced if there are visible signs of wear. Disinfection can be carried out using a commercially available immersion disinfectant. DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch Institute).

Handling the hand-mixing system

1. Remove equal amounts of the base paste using the colored spoon and the catalyst paste using the white spoon from the jars.
2. Knead the base and catalyst pastes at a ratio of 1:1 for 30 s at room temperature using the tips of the fingers, until the color is homogenous.

Note: Close the jars again immediately after use! In order to avoid premature setting, make sure that you do not switch the lids and spoons of the catalyst and base pastes or unintentionally mix the two pastes.

Note: Use gloves or foil to avoid contamination.

Recommended use

Double mixing impression

1. Fill the impression tray with Silagum Putty.
2. Inject around the preparations with Silagum Light.

Note: Loading of the impression tray and injection should be concluded at the same time, as the higher temperature in the mouth causes the injected material to set more quickly than the material in the impression tray

3. Position the filled impression tray slowly and gently in the mouth within the working time and take the impression of the preparation.
4. Remove the impression from the mouth once the setting time has ended (see “Removal of the impression”).

Corrective impression

Note: Do not allow more than 30 min to pass between completion of the initial impression and the corrective impression and do not clean the impression with alcohol, as there is then no guarantee of a secure bond between the two impression materials.

1. Fill the impression tray with Silagum Putty and insert into the mouth within the working time.
2. Maintain the position in accordance with the recommended intraoral setting time.
3. Remove the impression from the mouth once the setting time has ended.
4. Cut generous drainage channels into the initial impression and remove any imperfections.
5. After testing by repositioning, thoroughly rinse the impression with water and dry.
6. Apply Silagum Light around the preparation and onto the initial impression.
7. Position the filled impression tray in the mouth within the working time and exert brief pressure.
8. Remove the impression from the mouth once the setting time has ended (see “Removal of the impression”).

Removal of the impression

1. In the area where the teeth have been prepared, remove the impression parallel to the longitudinal axis of the teeth.
2. After removal, proceed as described in the section “Cleaning and disinfection of the final impression.”

Cleaning and disinfection of the final impression

1. Only disinfect the final impression using immersion solutions.
2. DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch Institute) or the EPA (Environmental Protection Agency). Observe the manufacturer’s instructions!
3. Ensure that the disinfectant solution used is indicated for use with addition-curing impression silicones.
4. Leave the final impression to dry and store until casting (see “Storage of the impression”).
5. Further information is also available in the DGZMK’s statement on the disinfection of dental impression materials. www.dgzmk.de

Casting and galvanization

- After removing the impression from the mouth, wait at least 30 min before casting.
- DMG recommends making the cast from hard modeling plaster (ISO 6873:2013; Type 3, e.g. Moldano, Bayer) and super-hard plaster for molds (ISO 6873:2013; Type 4, e.g. Fujirock II, GC).
- The impression can be copper or silver-plated.

Storing the impression

- Store the impression at 15 – 25 °C (59 – 77 °F) in a dry place!
- Protect from exposure to direct sunlight!

Timing

Silagum Putty			
	Fast	Standard	Soft
Mixing time [s]	30	30	30
Working time including mixing time* [s]	≤ 75	≤ 105	≤ 90
Setting time in the mouth [s]	≥ 180	≥ 225	≥ 225

Silagum Light		
	Fast	Standard
Working time including mixing time* [s]	≤ 90	≤ 120
Setting time in the mouth [s]	≥ 135	≥ 210

* The specified times are valid at a room temperature of 23 °C (73 °F) and a normal relative humidity of 50 %. Higher temperatures shorten, lower temperatures lengthen these times. The working time describes the period from when mixing begins until the latest point at which the tray must be positioned in the patient’s mouth.

Interactions

- Rinse off any residual retraction solution thoroughly with water, otherwise this may lengthen the setting time in some instances.
- Never allow the impression to come into contact with organic solvents or liquids containing solvents as this may result in a swelling of the material and may therefore lead to inaccuracies in the cast.
- Do not allow the impression material to come into contact with latex gloves. In the event of prior contact with latex gloves, wash hands carefully. DMG recommends the use of PVC or PE-based gloves.
- Do not combine the material with condensation-curing silicone or polyether impression materials.
- Residues of methacrylate-based materials may alter the setting behavior of the silicone impression materials. If a restoration was prepared beforehand, carefully clean the

preparation and the surrounding tissue with alcohol-soaked gauze and water spray before taking the impression.

Residual risks/side effects

- Please check the product for damage and visible contamination before use.
- Hypersensitivity and/or allergic reactions cannot be ruled out in individuals who are predisposed.
- Incorrect use of the product may lead to excessive loss of dental hard tissues or loss of prosthetic work, as well as respiratory tract irritation, irritation and/or injury to the skin or mucous membranes, accidental ingestion, patient discomfort and infections.
- A low residual risk of unknown side effects cannot be ruled out.

Warnings/precautions

- For dental use only!
- Avoid contact with eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products used with the material/materials.
- Observe the safety data sheet.
- The manufacturer (productsafety@mtcompanies.com) and the responsible registration authorities must be notified of any serious incidents involving this product.

Product performance characteristics

Meets the requirements of ISO 4823:2025; Silagum Putty: Type 0: putty consistency; Silagum Light: Type 3: light-bodied consistency

Storage/disposal

- Store at 15 – 25 °C (59 – 77 °F) in a dry place!
- Protect from exposure to direct sunlight!
- After use, clean the used mixing tip with a standard disinfectant solution and leave it on the Automix Cartridge to act as a cap!
- Do not use after the expiration date!
- Disposal must comply with national regulations.

Composition

Handmix Putty Standard: SiO₂, vinyl silicones/siloxanes, methyl silicones/siloxanes, hydrogen silicone, paraffins/vaseline, additives, pigments, catalysts.

Handmix Putty Soft: SiO₂, vinyl silicones/siloxanes, methyl silicones/siloxanes, additives, pigments, catalysts, hydrogen silicone.

Handmix Putty Fast: SiO₂, vinyl silicones/siloxanes, methyl silicones/siloxanes, additives, pigments, catalysts, hydrogen silicone.

Automix Light: vinyl silicones/siloxanes, methyl silicones/siloxanes, SiO₂, hydrogen silicone, additives, pigments, catalysts.

Automix Light Fast: vinyl silicones/siloxanes, methyl silicones/siloxanes, SiO₂, hydrogen silicone, additives, pigments, catalysts.

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