Silagum

Putty/Light

Inhalt | Contents

Gebrauchsinformation – DEUTSCH	4
Instructions for use – ENGLISH	12
Mode d'emploi – FRANÇAIS	21
Istruzioni d'uso – ITALIANO	30
Instrucciones de empleo – ESPAÑOL	39
Instruções de uso – PORTUGUÊS	48
Gebruiksaanwijzing – NEDERLANDS	57
Brugsanvisning – DANSK	65
Användarinstruktioner – SVENSKA	74
Instrukcja użycia – POLSKI	82
Информация по применению – РУССКИЙ	91
使用说明 – 中文	101
사용 방법 – 한국어	107

Produktbeschreibung

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Klassifikation

- Silagum-Putty: ISO 4823:2000, Type 0: Putty consistency
- Silagum-Light: ISO 4823:2000, Type 3: Light-bodied consistency

Handelsformen

Silagum-Putty

2 Dosen à 262 ml Paste	
Silagum-Putty Standard	REF 909018
Silagum-Putty Soft	REF 909019
Silagum-Putty Fast	REF 909037

8 Dosen à 262 ml Paste	
Silagum-Putty Standard	REF 909405
Silagum-Putty Soft	REF 909036
Silagum-Putty Fast	REF 909038

16 Dosen à 262 ml Paste	
Silagum-Putty Standard	REF 909420
Silagum-Putty Soft	REF 909485

Silagum-Light

2 Kartuschen à 50 ml Paste, 12 Automix-Tips, 12 Intraoral-Tips	
Silagum-Light	REF 909713
Silagum-Light Fast	REF 909714

8 Kartuschen à 50 ml Paste, 50 Automix-Tips, 50 Intraoral-Tips	
Silagum-Light	REF 909715
Silagum-Light Fast	REF 909720

8 Tuben à 90 ml (110 g) Paste, 2 Mischblöcke, 2 Anmischspatel	
Silagum-Handmix	REF 909422

Instructions for Use

English

Product description

Pre-impression material (Silagum-Putty) and correction impression material (Silagum-Light) based on A-silicones for all two-phase impression techniques. Silagum-Light is available in the Automix cartridge and as a hand-mixed variant.

Indications

Crown and bridge impressions, inlay and onlay impressions

Basic safety instructions

- ▶ For dental use only!
- ▶ Keep out of reach of children!

Interactions

- ▶ Avoid contact with retraction solutions, because in some instances residues of retraction solutions can increase the setting time.
- ▶ Never allow the impression to come into contact with organic solvents or liquids containing solvents as this may result in a swelling of the material and may therefore lead to inaccuracies in the cast.
- ▶ Avoid contact between the impression material and latex gloves because these inhibit the setting process. In the event of prior contact between the hands and latex gloves, wash hands thoroughly. It is recommended that PVC or PE based gloves be used.
- ▶ Do not mix Silagum with condensation curing silicone or polyether impression materials.

Using the Automix cartridge

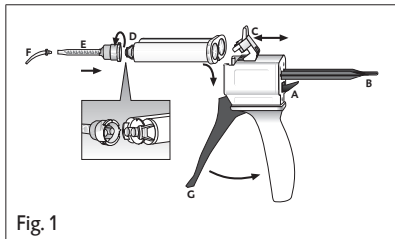
Fig. 1

Insert cartridge

1. Push up the lever [Fig. 1, Pos. A] at the back of the Automix-Dispenser and pull back the slide [Fig. 1, Pos. B] completely.
2. Lift up the plastic lever [Fig. 1, Pos. C], insert the cartridge and lock it using the plastic lever [Fig. 1, Pos. C].

Note: Make sure that the notches on the cartridge line up with those on the Automix-Dispenser.

Attach mixing tip

1. Remove the cartridge cap [Fig. 1, Pos. D] or used mixing tip by turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [Fig. 1, Pos. E].

Note: In order to attain an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are also available as refill packs.

Note: Make sure that the notches on the mixing tip line up with those on the cartridge.

3. Turn the mixing tip clockwise 90° to lock it.
4. Attach an intraoral tip [Fig. 1, Pos. F].

Applying the material

- ▶ When using a cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- ▶ The material is mixed in the mixing tip by pulling the trigger [Fig. 1, Pos. G] of the Automix dispenser and it is then applied directly.

Note: After the application, leave the used mixing tip on the cartridge to cap unused material!

Remove cartridge

1. Push up the lever [Fig. 1, Pos. A] at the back of the Automix-Dispenser and pull back the slide [Fig. 1, Pos. B] completely.
2. Lift up the plastic lever [Fig. 1, Pos. C] and remove the cartridge.

Handling the hand-mixing system

Can

- ▶ For the individual times, refer to the “Technical Data” chapter.
1. Take equal measures of the base and the catalyst pastes from the can using the colored spoon and the white spoon respectively.
 2. Knead the base and catalyst paste at a ratio of 1:1 for 30 s at room temperature using the tips of the fingers, until the color is homogeneous.

Note: Close the can again immediately after use! Make sure that you do not switch the lids and spoons of the catalyst and base pastes or unintentionally mix the two pastes, in order to avoid premature setting.

Tube

1. Measure out equal lengths of catalyst and base paste (1:1) at room temperature.
2. Mix the catalyst and base paste for 30 s on a suitable mixing surface (e.g. a sheet of glass, mixing block) using a plastic or metal spatula until the color is homogenous.
3. Fill the mixed material into an elastomer syringe for application.

Note: Close the tube again immediately after use! Make sure that you do not switch the lids of the catalyst and base pastes and that you do not unintentionally mix the two pastes, in order to avoid blocking the tube openings caused by cross contamination.

Instructions for use

- ▶ Silagum-Putty can be used in all common impression trays. In order to improve the adhesion of the material to metal or plastic trays, a tray adhesive can be applied (e.g. DMG Tray Adhesive).
- ▶ In order to ensure that the two impression materials bond securely with each other, a period of no more than 30 min may pass between the pre-impression and correction impression. Alternatively, it is possible to employ the double mixing impression technique.
- ▶ For the individual times, refer to the “Technical Data” chapter.

Recommended use

Double-mix impressions

1. Fill the tray with Silagum-Putty.
2. Syringe the preparations with Silagum-Light.

Note: The filling of the impression tray and syringing should be completed at the same time, because

the higher temperature in the mouth leads to a quicker setting of the syringed material compared to the tray material.

3. Position the filled impression tray in the mouth and allow the impression to form.
4. Remove the impression from the mouth once the setting time has ended.

Correction impression

1. Fill the tray with Silagum-Putty and insert into the mouth within the working time. Maintain this position in accordance with the recommended intraoral setting time.
2. Remove the impression from the mouth once the setting time has ended.
3. Cut generous drainage channels into the pre-impression for the wash and remove imperfections.
4. After repositioning, thoroughly rinse the impression with water and dry.
5. Apply Silagum-Light around the preparation and on the pre-impression.

6. Position the tray in the mouth within the working time and initially exert brief pressure.
7. Remove the impression from the mouth once the setting time has ended.

Removal of the impression

- ▶ Remove the impression parallel to the longitudinal axis of the prepared teeth.
- ▶ After removal, proceed as described in the section “Cleaning and disinfection of the final impression”.

Cleaning and disinfection of the final impression

- ▶ Only disinfect the final impression using dipping solutions. DMG recommends the use of a 2 % glutaraldehyde solution.
- ▶ Please ensure that the disinfectant used is indicated for use with addition curing silicone impression materials.

- ▶ Follow the instructions of the disinfectant solution's manufacturer precisely.
- ▶ Leave the final impression to dry and store until casting, see "Storage of the impression".
- ▶ For more information, refer to the comments of the DGZMK. Comments of the DGZMK 3/93 V 1.0; www.dgzmk.de
- ▶ ADA recommends following Council on Dental Materials, Instruments and Equipment. Disinfection of impressions. JADA 1991;122(3):110; www.ada.org

Casting and galvanization

- ▶ The impression should be cast 30 minutes at the earliest after removal from the mouth. DMG recommends using special purpose hard plaster (ISO 6873; Type 3, e.g. Moldano, Bayer) for casting and an ultra hard dental plaster for the forms (ISO 6873; Type 4, e.g. Fujirock II, GC). The impression can be copper-plated or silver-plated.

Storage of the impression

- ▶ Store impression at room temperature (15 – 25 °C/59 – 77 °F) in a dry place!
 - ▶ Protect against exposure to direct sunlight!
- The impression is stable and can be cast even after months without loss of accuracy.

Disinfection of the Automix dispenser

The Automix dispenser is reusable after disinfection, although it should be replaced if visible signs of wear appear.

Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute). The Automix dispenser is also autoclavable.

Technical Data

Silagum-Putty	Fast	Standard	Soft
Mixing time	0:30 min	0:30 min	0:30 min
Processing time* including mixing time	≤ 1:15 min	≤ 1:45 min	≤ 1:45 min
Recommended time in mouth	≥ 2:30 min	≥ 3:30 min	≥ 3:30 min
Dimensional change (24h)	≤ 0.5%	≤ 0.5%	≤ 0.5%
Compression set	≤ 0.7%	≤ 0.7%	≤ 0.7%

Silagum-Putty	Fast	Standard	Soft
Strain in compression	≈ 1.7%	≈ 1.7%	≈ 2.5%

Silagum-Light	Fast	Standard
Mixing time**	-	0:30 min
Processing time* including mixing time	≤ 1:30 min	≤ 2:15 min
Recommended time in mouth	≥ 2:15 min	≥ 3:30 min
Dimensional change (24h)	≤ 0.4%	≤ 0.4%
Compression set	≤ 0.4%	≤ 0.4%
Strain in compression	≈ 3.5%	≈ 3.5%

*The times apply at a room temperature of 23 °C and a normal relative air humidity of 50 %. Higher temperatures shorten, lower temperatures increase these times.

**The mixing time only applies to the hand-mixed variant.

Storage and shelf-life

- ▶ Store at room temperature (15 – 25 °C/59 – 77 °F) in a dry place.
- ▶ Do not store in the fridge or freezer.
- ▶ Protect against exposure to direct sunlight.
- ▶ After the application, leave the used tip on the cartridge to cap unused material.
- ▶ Close the can again immediately after use.
- ▶ Do not use after the expiration date.

Ingredients

Addition curing vinyl polysiloxanes, hydrogen polysiloxanes, fillers, pigments, additives, platinum catalyst

Classification

- Silagum-Putty: ISO 4823:2000, type 0: Putty consistency
- Silagum-Light: ISO 4823:2000, type 3: Light-bodied consistency

Available product sizes

Silagum-Putty

2 cans @ 262 ml paste	
Silagum-Putty Standard	REF 909018
Silagum-Putty Soft	REF 909019
Silagum-Putty Fast	REF 909037

8 cans @ 262 ml paste	
Silagum-Putty Standard	REF 909405
Silagum-Putty Soft	REF 909036
Silagum-Putty Fast	REF 909038

16 cans @ 262 ml paste	
Silagum-Putty Standard	REF 909420
Silagum-Putty Soft	REF 909485

Federal law (USA) restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

Silagum-Light

2 cartridges @ 50 ml paste, 12 Automix tips, 12 Intraoral tips	
Silagum-Light	REF 909713
Silagum-Light Fast	REF 909714

8 cartridges @ 50 ml paste, 50 Automix tips, 50 Intraoral tips	
Silagum-Light	REF 909715
Silagum-Light Fast	REF 909720

8 tubes @ 90 ml (110 g) paste, 2 mixing blocks, 2 mixing spatulas	
Silagum-Handmix	REF 909422

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DMG Chemisch-Pharmazeutische Fabrik GmbH