Silagum Medium

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Instructions for use English

Product description

Medium-viscosity impression material based on A-silicones. The material can be used as a correction impression material for all two-phase impression techniques. As an extrudable monophasic impression material, Silagum Medium can be used in partial and complete denture prosthetics, e.g. for reline impressions, functional impressions, impressions for prosthetics repairs. Silagum Medium is available in the Automix Cartridge.

Intended purpose

- Impressions for the fabrication of indirect restorations, such as crowns, bridges, inlays and onlays.
- Reline and functional impressions
- Pressure-spot indication

Limitations of use

No limitations of use.

Indication

All dental situations requiring final impressions, e.g. tooth loss, caries and trauma.

Contraindication

Do not use the material if there are known allergies to any of the components or contact allergies.

Patient target group

People to be treated in the course of a dental procedure.

Intended user

Dentist, dental assistant staff

Notes for use

- The recommended times can be found in the section headed "Timing".
- If the material is applied intraorally using a single-use applicator, the single-use applicator must be used only on this one patient for hygienic reasons.
- To achieve an optimum mixing result, use the mixing tips available from DMG. All mixing tips are available as refill packs.
- The Intraoral Tips are to be used for direct, intrasulcular application around the preparation.
- In order to improve the adhesion of the initial impression material to metal or plastic trays, a tray adhesive for A-silicones should be applied (e.g. DMG Tray Adhesive).

Using the Automix Cartridge (Fig. see "Handling")

Using the Automix Cartridge

- Push up the lever [A] at the back of the dispensing gun and pull back the slide [B] completely.
- Lift up the plastic lever [C], insert the Automix Cartridge and lock it using the plastic lever [C].

Note: Make sure that the notches on the Automix Cartridge line up with those on the dispensing unit.

Attaching a mixing tip

- Remove the cartridge cap [D] or used mixing tip after turning it 90° counterclockwise, and discard it.
- Attach a new mixing tip [E].

Note: Make sure that the notches on the mixing tip line up with those on the Automix Cartridge.

3. Twist the mixing tip clockwise by 90° to lock it.
4. Attach an Intraoral Tip [F], if needed.

Applying the material

- When using an Automix Cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- Mix the material in the mixing tip using the lever [G] on the dispensing gun and apply directly.

Note: After the application, leave the used mixing tip on the Automix Cartridge as a seal!

Removing the Automix Cartridge

1. Push up the lever [A] at the back of the dispensing gun and pull back the slide [B] completely.
2. Lift up the plastic lever [C] and remove the cartridge.

Disinfecting the Automix Dispenser

The Automix Dispenser can be reused after disinfection, although it should be replaced if there are visible signs of wear. Disinfection can be carried out using a commercially available immersion disinfectant. DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch Institute).

Recommended use

Monophase impressions

1. Fill the prepared tray with Silagum Medium. When doing so, hold the impression tray at a slight angle beneath the mixing tip.

Note: To avoid air bubbles, the end of the mixing tip should always remain immersed in the material while filling the tray.

2. Inject around the preparation with an Intraoral Tip.

Note: Filling of the impression tray and injection should be concluded at the same time, as the higher temperature in the mouth causes the injected material to set more quickly than the material in the impression tray.

3. Position the filled impression tray slowly and gently in the mouth within the working time and take the impression of the preparation.
4. Remove the impression from the mouth once the setting time has ended (see "Removal").

Corrective impression

Note: Do not allow more than 30 min to pass between completion of the initial impression and the final impression and do not clean the impression with alcohol, as there is then no guarantee of a secure bond between the two impression materials.

1. Cut generous drainage channels into the initial impression (e.g. made with Silagum Putty) for the wash and remove imperfections.
2. After testing by repositioning, rinse the impression thoroughly with water and then dry.
3. Apply Silagum Medium around the preparation and onto the initial impression.
4. Position the filled impression tray in the mouth within the working time, initially exerting brief pressure.
5. Remove the impression from the mouth once the setting time has ended (see "Removal").

Removal

- In the area where the teeth have been prepared, remove the impression parallel to the longitudinal axis of the teeth.
- After removal, proceed as described in the section "Cleaning and disinfection of the final impression."

Cleaning and disinfection of the final impression

- Only disinfect the final impression using immersion solutions.
- DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch Institute) or the EPA (Environmental Protection Agency). Observe the manufacturer's instructions!
- Ensure that the disinfectant used is indicated for use with addition-cured silicone impression material.
- Leave the final impression to dry and store until casting (see "Storage of the impression").
- Further information is also available in the DGZMK's statement on the disinfection of dental impression materials: www.dgzmk.de.

Casting and galvanization

- After removing the impression from the mouth, wait at least 30 min before casting.
- DMG recommends making the cast from a special purpose hard plaster (ISO 6873:2013; Type 3, e.g. Moldano, Bayer) and an ultra-hard dental plaster for the models (ISO 6873:2013; Type 4, e.g. Fujirock II, GC).
- The impression can be copper or silver-plated.

Storing the impression

- Store the impression at 15 – 25 °C (59 – 77 °F) in a dry place!
- Protect from exposure to direct sunlight!

Timing

Silagum Medium		
	Standard	Fast
Working time including mixing time* [s]	≤ 135	≤ 90
Setting time in the mouth [s]	≥ 210	≥ 135

* The specified times are valid at a room temperature of 23 °C (73 °F) and a normal relative humidity of 50 %. Higher temperatures shorten, lower temperatures lengthen these times. The working time describes the period from when mixing begins to the time by which the impression tray must be positioned in the patient's mouth.

Interactions

- Rinse off any residual retraction solution thoroughly with water, otherwise this may lengthen the setting time in some instances.
- Never allow the impression to come into contact with organic solvents or liquids containing solvents as this may result in a swelling of the material and may therefore lead to inaccuracies in the cast.
- Do not allow the impression material to come into contact with latex gloves. In the event of prior contact with latex gloves, wash hands carefully. DMG recommends the use of PVC or PE-based gloves.
- Do not mix Silagum Medium with condensation curing silicone or polyether impression materials.
- Residues of methacrylate-based materials may alter the setting behavior of the silicone impression materials. If a restoration was prepared beforehand, carefully clean the preparation and the surrounding tissue with alcohol-soaked gauze and water spray before taking the impression.

Residual risks/side effects

- Please check the product for damage and visible contamination before use.
- A low residual risk of unknown side effects cannot be ruled out
- Hypersensitivity and/or allergic reactions cannot be ruled out in individuals who are predisposed.

Warnings/precautions

- For dental use only!
- Avoid contact with eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products used with the material/materials.
- Observe the safety data sheet.
- Serious incidents involving this product must be reported to the manufacturer (productsafety@mtcompanies.com).

Product performance characteristics

Meets the requirements of ISO 4823:2025, Type 2: medium-bodied consistency

Storage/disposal

- Store at 15 – 25 °C (59 – 77 °F) in a dry place.
- Protect from exposure to direct sunlight!
- After the application, leave the used mixing tip on the Automix Cartridge as a seal!
- Do not use after the expiration date!
- Disposal must comply with national regulations.

Composition

Automix: Vinyl silicones/siloxanes, SiO₂, paraffin oil, methyl silicones/siloxanes, additives
Automix Fast: Vinyl silicones/siloxanes, SiO₂, paraffin oil, methyl silicones/siloxanes, additives.

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