Silagum

MixStar Putty Soft

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Vinylpolysiloxane, Hydrogenpolysiloxane, Füllstoffe, Pigmente, Additive, Platinkatalysator

Klassifikation

ISO 4823:2000, Type 1: Heavy bodied consistency

Handelsformen

1 Kartusche à 380 ml Paste, 10 Mischkanülen	REF 909534
5 Kartuschen à 380 ml Paste, 50 Mischkanülen	REF 909535

Instructions for Use

English

Product description

Pre-impression material based on A-silicones. Silagum-MixStar Putty Soft is available in the MixStar cartridge, which is suitable for use in the DMG MixStar devices and other commercially available mixing machines.

Indications

- Crown and bridge impression, inlay and onlay impressions

Basic safety instructions

- For dental use only!
- Keep out of reach of children!

Interactions

Never allow the impression to come into contact with organic solvents or liquids containing solvents as this may result in a swelling of the material and may therefore lead to inaccuracies in the cast.

Avoid contact with retraction solutions, because in some instances residues of retraction solutions can increase the setting time.

Do not mix Silagum with condensation curing silicone or polyether impression materials.

Avoid contact between the impression material and latex gloves because these prevent the setting process. In the event of prior contact between the hands and latex gloves, wash hands thoroughly. It is recommended that PVC or PE based gloves be used.

Using the MixStar cartridge in the DMG MixStar-eMotion

Follow the instructions for use for the "MixStar-eMotion"!

- Slide the locking mechanism upwards in order to open the cartridge. Do not remove the locking mechanism from the cartridge (Fig. 1).
- Remove the cap from the cartridge and discard. Attach a new mixing tip (Fig. 2).
- Secure the mixing tip by sliding the locking mechanism into the locked position (Fig. 3).
- Insert the cartridge into the MixStar-eMotion (Fig. 4). A brief acoustic signal confirms the correct insertion of the cartridge.

Note: Prepare fresh cartridges for use by extruding and discarding the first 3 cm of impression material.

Note: After the application, leave the used tip on the cartridge to cap unused material.

Use in other mixing machines

Follow the instructions of the mixing unit you are using!

First insert the cartridge into the mixing unit without the mixing tip. Otherwise appropriate mixing of the material cannot be guaranteed!

In this case proceed as described in the section "Changing the mixing tip".

Changing the mixing tip

Follow the instructions of the mixing unit you are using!

Note: Attach a new mixing tip before each mixing process.

- Slide the locking mechanism upwards.
- Remove the old mixing tip and any hardened material blocking the cartridge openings.
- Place the mixing tip securely on the respective cartridge opening.
- Lock the mixing tip in place with the locking mechanism.

Instructions for use

Silagum-MixStar Putty Soft can be used in all conventional impression trays. In order to improve the adhesion to metal or plastic trays, a tray adhesive can be applied.

In order to ensure that the two impression materials bond with each other, a period of no more than 30 minutes may pass between the pre-impression and correction impression. Alternatively, it is possible to employ the double mixing impression technique.

Recommended use

Double-mix impressions

- Fill the tray with Silagum-MixStar Putty Soft. While the assistant fills the tray the dentist can syringe the preparation, e.g. with Silagum-Light.

Note: The filling of the impression tray and syringing should be completed at the same time, because the higher temperature in the mouth leads to a quicker setting of the syringed material compared to the tray material.

- Position the filled impression tray in the mouth and allow the impression to form.
- Remove the impression from the mouth once the setting time has ended.

Correction impression

1. Fill the prepared impression tray with Silagum-MixStar Putty Soft. When doing so, hold the impression tray at a slight angle beneath the mixing tip.

Note: To avoid air bubbles, the end of the mixing tip should stay immersed in the impression material throughout the application process.

2. Insert the tray in the mouth within the working time. Maintain this position in accordance with the recommended intraoral setting time.
3. Remove the impression from the mouth once the setting time has ended.
4. Cut generous drainage channels into the pre-impression for the wash and remove imperfections.
5. After repositioning, thoroughly rinse the impression with water and dry.
6. Apply Silagum-Light around the preparation and on the pre-impression.
7. Position the filled tray in the mouth within the working time and initially fix in place by applying even pressure.
8. Remove the impression from the mouth once the setting time has ended.

Removal of the impression

Remove the impression parallel to the longitudinal axis of the prepared teeth. After removal, proceed as described in the section "Cleaning and disinfection of the final impression".

Cleaning and disinfection of the final impression

Only disinfect the final impression using dipping solutions. DMG recommends the use of a 2% glutaraldehyde solution.

Please ensure that the disinfectant used is indicated for use with addition curing silicone impression materials.

Follow the instructions of the disinfectant solution's manufacturer precisely.

Leave the final impression to dry and store until casting, see "Storage of the impression".

For more information, refer to the comments of the DGZMK. Comments of the DGZMK 3/93 V 1.0; www.dgzmk.de.

ADA recommends following Council on Dental Materials, Instruments and Equipment. Disinfection of impressions. JADA 1991;122(3):110; www.ada.org

Casting and galvanization

The impression should be cast 30 minutes at the earliest after removal from the mouth.

DMG recommends using special purpose hard plaster (ISO 6873; Type 3, e.g. Moldano, Bayer) for the cast and an ultra hard dental plaster for the forms (ISO 6873; Type 4, e.g. Fujirock II, GC). The impression can be copper-plated or silver-plated.

Storage of the impression

Store impression at room temperature (15 – 25 °C/59 – 77 °F) in a dry place! Protect against exposure to direct sunlight!

The impression is stable and can be cast even after months without loss of accuracy.

Technical data

Working time*	≤ 2:00 min
Recommended time in mouth	≥ 3:30 min
Dimensional change after 24 h	≤ 0.5 %
Compression set	≤ 0.5 %
Strain in compression	1.7 – 2.6 %

*The times apply at a room temperature of 23 °C and a normal relative air humidity of 50%. Higher temperatures shorten, lower temperatures increase this time.

Storage and shelf life

Store at room temperature (15 – 25 °C/59 – 77 °F) in a dry place.

Do not store in the fridge or freezer.

Protect against exposure to direct sunlight.

After the application, leave the used tip on the cartridge to cap unused material.

Do not use after the expiration date.

Composition

Vinyl polysiloxanes, hydrogen polysiloxanes, fillers, pigments, additives, platinum catalyst

Classification

ISO 4823:2000, type 1: Heavy bodied consistency

Available product sizes

1 cartridge @ 380 ml paste, 10 mixing tips	REF 909534
5 cartridges @ 380 ml paste, 50 mixing tips	REF 909535

Federal law (USA) restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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