Silagum Putty Soft

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English

Product description

Initial impression material based on A-silicones. Silagum Putty Soft is available in the MixStar Cartridge, which is suitable for use in the DMG MixStar devices and other commercially available mixing machines.

Intended purpose

Impressions for the fabrication of indirect restorations, such as crowns, bridges, inlays and onlays.

Limitations of use

Do not use the material for monophasic impressions.

Indication

All dental situations requiring final impressions, e.g. tooth loss, caries and trauma.

Contraindication

Do not use the material if there are known allergies to any of the components or contact allergies.

Patient target group

People who are receiving dental treatment.

Intended user

Dentist, dental assistant staff

Notes for use

- The recommended times can be found in the section headed "Timing."
- To achieve an optimum mixing result, use the mixing tips available from DMG. All mixing tips are also available in refill packs.
- Silagum Putty Soft can be used in all conventional impression trays.
- In order to improve the adhesion of the material to metal or plastic trays, a tray adhesive can be applied.

Using the MixStar Cartridge in the MixStar eMotion (Fig. see "Handling")

Follow the instructions for use for the "MixStar eMotion" unit!

- Slide the locking mechanism upwards in order to open the MixStar Cartridge. Do not remove the locking mechanism from the MixStar Cartridge [Fig. 1].
- Remove the seal from the MixStar Cartridge and discard. Attach a new mixing tip [Fig. 2].
- Secure the mixing tip by sliding the locking mechanism into the locked position [Fig. 3].
- Insert the MixStar Cartridge into the MixStar eMotion [Fig. 4]. A brief acoustic signal confirms correct insertion of the cartridge.

Note: Prior to using a new cartridge for the first time, discard approx. the first 3 cm of the ejected material.

Note: After use, clean the used mixing tip with a standard disinfectant solution and leave it on the MixStar Cartridge to act as a cap!

Using the MixStar Cartridge in mixing units from other manufacturers

Follow the instructions for use for the mixing unit used!

- First insert the MixStar Cartridge into the mixing unit without the mixing tip. Otherwise appropriate mixing of the material cannot be guaranteed!
- In this case proceed as described in the section “Changing the mixing tip.”

Changing the mixing tip

Follow the instructions for use for the mixing unit used!

Note: Attach a new mixing tip each time before mixing.

1. Slide the locking mechanism upwards.
2. Remove the old mixing tip as well as any hardened material blocking the cartridge openings.
3. Place the mixing tip securely on the corresponding MixStar Cartridge opening.
4. Lock the mixing tip in place with the locking mechanism.

A list of validated devices can be found on our website (Accessories—MixStar eMotion—Downloads).

Recommended use

Double mixing impression

1. Fill the impression tray with Silagum Putty Soft. When doing so, hold the impression tray at a slight angle beneath the mixing tip.

Note: To avoid air bubbles, the end of the mixing tip should always remain immersed in the material while filling the tray.

2. Inject around the preparation with Silagum Light. Follow the corresponding instructions for use!

Note: Filling of the impression tray and injection should be concluded at the same time, as the higher temperature in the mouth causes the injected material to set more quickly than the material in the impression tray.

3. Position the filled impression tray slowly and gently in the mouth within the working time and take the impression of the preparation.
4. Remove the impression from the mouth once the setting time has ended (see “Removal”).

Corrective impression

Note: Do not allow more than 30 min to pass between completion of the initial impression and the wash impression and do not clean the impression with alcohol, as there is then no guarantee of a secure bond between the two impression materials.

1. Fill the prepared impression tray with Silagum Putty Soft. When doing so, hold the impression tray at a slight angle beneath the mixing tip.

Note: To avoid air bubbles, the end of the mixing tip should always remain immersed in the material while filling the tray.

2. Insert the tray in the mouth within the working time. Maintain this position in accordance with the recommended intraoral setting time.
3. Remove the impression from the mouth once the setting time has ended.
4. Cut generous drainage channels into the initial impression and remove any imperfections.
5. After testing by repositioning, rinse the impression thoroughly with water and dry.
6. Apply Silagum Light around the preparation and onto the initial impression. Follow the corresponding instructions for use!
7. Position the filled impression tray slowly and gently in the mouth within the working time, initially exerting brief pressure.
8. Remove the impression from the mouth once the setting time has ended (see “Removal”).

Removal of the impression

- In the area where the teeth have been prepared, remove the impression parallel to the longitudinal axis of the teeth.
- After removal, proceed as described in the section “Cleaning and disinfection of the final impression.”

Cleaning and disinfection of the final impression

- Only disinfect the final impression using immersion solutions.
- DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch Institute) or the EPA (Environmental Protection Agency). Observe the manufacturer’s instructions!
- Ensure that the disinfectant solution used is indicated for use with addition-curing impression silicones.
- Leave the final impression to dry and store until casting (see “Storage of the impression”).
- Further information is also available in the DGZMK’s statement on the disinfection of dental impression materials. www.dgzmk.de

Casting and galvanization

- After removing the impression from the mouth, wait at least 30 min before casting.
- DMG recommends making the cast from hard modeling plaster (ISO 6873:2013; Type 3, e.g. Moldano, Bayer) and super-hard plaster for molds (ISO 6873:2013; Type 4, e.g. Fujirock II, GC).
- The impression can be copper or silver-plated.

Storing the impression

- Store the impression at 15 – 25 °C (59 – 77 °F) in a dry place!
- Protect from exposure to direct sunlight!

Timing

Silagum Putty Soft	
Working time including mixing time* [s]	≤ 120
Setting time in the mouth [s]	≥ 210

* The specified times are valid at a room temperature of 23 °C (73 °F) and a normal relative humidity of 50 %. Higher temperatures shorten, lower temperatures lengthen these times. The working time describes the period from when mixing begins to the time by which the impression tray must be positioned in the patient’s mouth.

Interactions

- Rinse off any residual retraction solution thoroughly with water, otherwise this may lengthen the setting time in some instances.
- Never allow the impression to come into contact with organic solvents or liquids containing solvents as this may result in a swelling of the material and may therefore lead to inaccuracies in the cast.
- Do not allow the impression material to come into contact with latex gloves. In the event of prior contact with latex gloves, wash hands carefully. DMG recommends the use of PVC or PE-based gloves.
- Do not combine the material with condensation-curing silicone or polyether impression materials.
- Residues of methacrylate-based materials may alter the setting behavior of the silicone impression materials. If a restoration was prepared beforehand, carefully clean the preparation and the surrounding tissue with alcohol-soaked gauze and water spray before taking the impression.

Residual risks/side effects

- Please check the product for damage and visible contamination before use.
- Hypersensitivity and/or allergic reactions cannot be ruled out in individuals who are predisposed.
- A low residual risk of unknown side effects cannot be ruled out.

Warnings/precautions

- For dental use only!
- Avoid contact with eyes! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer’s instructions for other products used with the material/materials.
- Observe the safety data sheet.
- The manufacturer (productsafety@mtcompanies.com) and the responsible registration authorities must be notified of any serious incidents involving this product.

Product performance characteristics

Meets the requirements of ISO 4823:2025, Type 1: heavy-bodied consistency

Storage/disposal

- Store in a dry place at 15 to 25 °C (59 to 77 °F)!
- Protect from exposure to direct sunlight!
- After use, clean the used mixing tip with a standard disinfectant solution and leave it on the MixStar Cartridge to act as a cap!
- Do not use after the expiration date!
- Disposal must comply with national regulations.

Composition

Cristobalite, polyvinylsiloxane, aluminosilicate (hydrated), SiO₂, additives, pigments, catalysts

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