Silagum-Comfort

Primer & Varnish

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English

Product description

Silagum-Comfort Primer is a bonding agent for Silagum-Comfort.
Silagum-Comfort Varnish is used for post-processing of a prosthesis relined with Silagum-Comfort to create an even surface.

Indications

- Silagum-Comfort Primer: Bonding between standard PMMA-based prosthesis materials and Silagum-Comfort
- Silagum-Comfort Varnish: Post-processing of a prosthesis relined with Silagum-Comfort

Basic safety instructions

For dental use only!
Keep out of reach of children!

Recommended use

Applying Silagum-Comfort Primer

1. Apply Silagum-Comfort Primer to the surface of the prepared prosthesis and allow the solvent to completely evaporate (approx. 1 min). Ensure that the primer is applied thoroughly to the areas at the edges. Apply at least two layers.
2. Close the bottle tightly to prevent the solvent from evaporating.

Applying Silagum-Comfort Varnish

1. Mix Silagum-Comfort Varnish evenly at a ratio of 1:1.
2. Using a brush, apply Silagum-Comfort Varnish to the prepared areas of the prosthesis within the working time, avoiding air bubbles as much as possible.
3. Allow Silagum-Comfort Varnish to dry for 15 min.

Technical data

| | Silagum-Comfort Primer | Silagum-Comfort Varnish |
|-----------------|------------------------|-------------------------|
| Working time* | - | ≤ 2 min |
| Hardening time* | - | ≥ 15 min |
| Airing time | ≥ 1 min | - |

* The times apply at a room temperature of 23 °C and a normal relative air humidity of 50%. Higher temperatures shorten, lower temperatures increase these times.

Disinfection

The brush holder is reusable after disinfection, although it should be replaced if visible signs of wear, e.g. scratches, appear. Disinfection can be carried out using a commercially available wiping or dipping disinfectant. DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute).

Storage and shelf life

Store at room temperature (15 – 25 °C/59 – 77 °F) in a dry place.
Do not store in the fridge or freezer.
Protect against exposure to direct sunlight.
Do not use after the expiration date.

Composition

Silagum-Comfort Primer: Ethyl acetate, modified polyacrylate, additive
Silagum-Comfort Varnish: Vinyl silicone, hydrogen silicone, additive, platinum catalyst

Available product sizes

| | |
|---|------------|
| Silagum-Comfort Primer [1 bottle @ 5 ml] | REF 909090 |
| Silagum-Comfort Varnish [2 bottles @ 10 ml] | REF 909089 |

Federal law (USA) restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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Application du Silagum-Comfort Varnish

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