



Infiltration with Icon smooth surface: micro-invasive method with a wide range of indications.



More information on diagnosis and treatment steps can be found on the following page and at www.dmg-dental.com/icon-vestibular



DENTAL HISTORY AND DIAGNOSIS

White Spots

Fluorosis

Trauma

MIH*

Diagnosis depth of the spot

Superficial lesion

Mixed lesion

Deep lesion

PRE-TREATMENT

Bleaching (if necessary)

Waiting for 2 weeks

14

Rubber Dam/ Liquid Dam

Microabrasion (optional)

Sandblasting/Microabrasion

Diamond Bur

INFILTRATION

Icon-Etch

2 min

Repeat

Spots diminished

Icon-Dry

30 sec

Spots are still visible

≥3 min

Icon-Infiltrant

Light-curing

40 sec

40 sec

Light-curing

Icon-Infiltrant

1 min

Add Composite (optional)

Polishing

* MIH is not indicated in the manufacturer's instruction for use.